

GMP
ISO9001
ISO14001
ISO13485
CERTIFIED

PIHSIANG®

SHOPRIDER®

MODEL: **TE-779XLS**



(PATENTED)

PIHSIANG MACHINERY MFG. CO., LTD.


NO.108, HSIN-HE ROAD, HSIN-FENG HSIANG 304
HSINCHU COUNTY, TAIWAN

MODEL :



[illegible]

TABLE OF CONTENTS
ILLUSTRATION

INDEX.1	CHASSIS ASSEMBLY -----	1-1~2
INDEX.2	FRONT AXLE ASSEMBLY -----	2-1
INDEX.3	DELUXE SEAT ASSEMBLY -----	3-1
INDEX.4	FRONT CHASSIS COVER ASSEMBLY -----	4-1
INDEX.5	REAR CHASSIS COVER ASSEMBLY -----	5-1
INDEX.6	HEAD LIGHT ASSEMBLY -----	6-1
INDEX.7	CONTROL AND BATTERY WIRING ILLUSTRATION -----	7-1
	APPENDIX - STICKER AND DECAL -----	

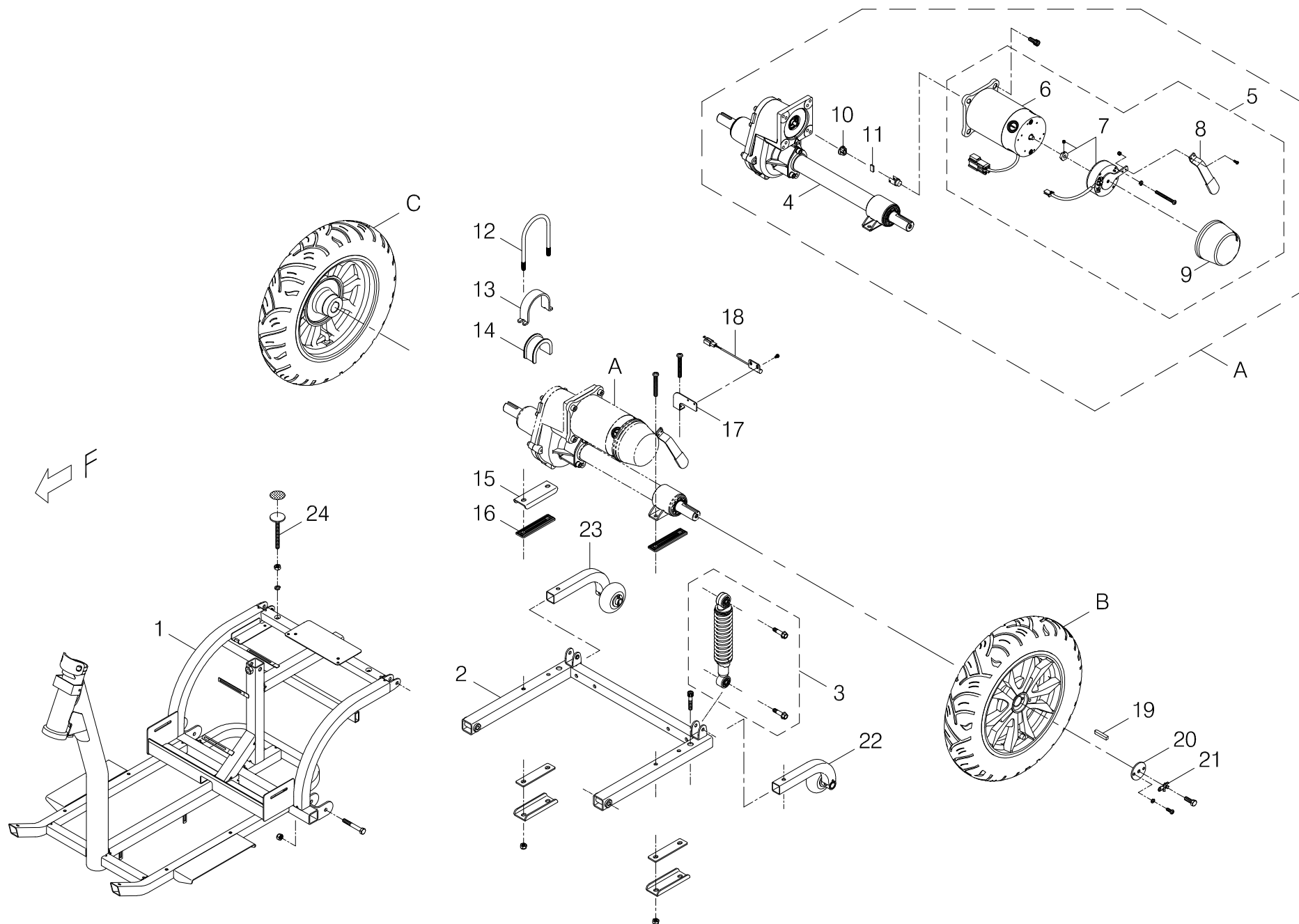

Index NO.	Part NO.	Qty.	Description	Specification	Remark
1	-----		Chassis assembly		
1-1	102101-72500	1	Front Chassis	779XLS	
1-2	102203-72500	1	Rear Chassis Assembly	Lower, 779XLS	
1-3	202931-89100	2	Adjustable Shock Absorber	w/ Screws, 5-Stage	
			-----*-----		
1-A	100008-67212	1	Drive-train Assembly	9XL-4G+#8C+#6A(A380-28)	
1-4	100406-68201-A4	1	Transaxle Assembly	#6A(C), 889X(624TL), A4	
1-5	100107-67207	1	Motor and Brake Assembly	9XL-4G+#8C(A380-28)	
1-6	100204-03011-A2	1	Motor	PH-9XL-4G, 5450rpm, A2	
-----	200203-89102-A2	REF(4)	Carbon Brush	9XL,7*17*17,A2	
-----	200203-66907	REF(4)	Carbon Brush Cap	9XL,M20	
1-7	100303-71600	1	Electromagnetic Brake Assembly	#8C, WR218A34P1, w/ UL7-W-L1 Cable	
1-8	4001380-28	1	Extension Lever for Free-wheel	A380-28	
1-9	209929-67205	1	Rubber Cap	Black, for #8 Brake-DN001	
1-10	200904-88100	1	Joint Adapter	for Transaxle	
1-11	200905-88100	1	Plate for Joint Adapter		
1-12	4003210-07	1	U-type Screw	C210-7	
1-13	4001530-00	1	Bracket	A530	
1-14	200912-00400	1	rubber collar	889DXN	
1-15	4001168-02	3	Bracket	A168-2	
1-16	209909-68201	4	Rubber Gasket w/ 2 Holes	106*26*4.75t	
1-17	4001236-27	1	Speed Sensor Bracket	A236-27	
1-18	206802-71501	1	Speed Sensor	S Type Sensor 100cm	
			-----*-----		

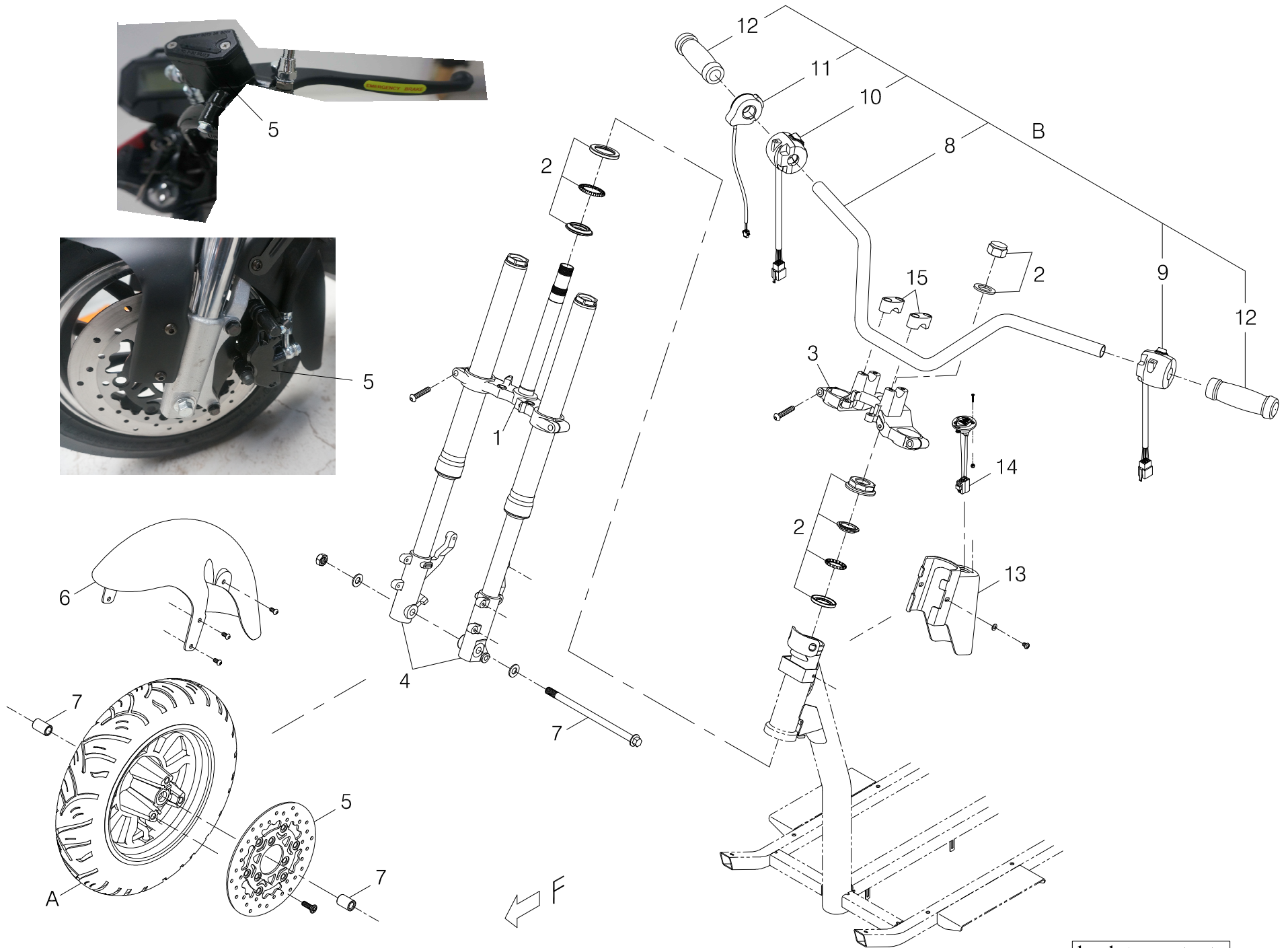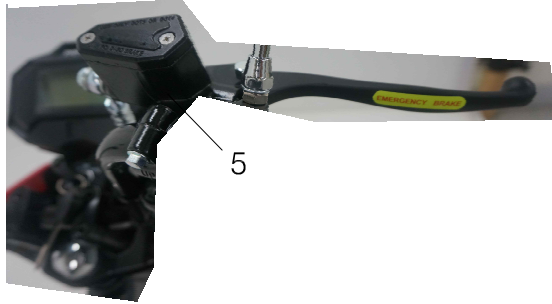



Index NO.	Part NO.	Qty.	Description	Specification	Remark
2	-----		Front axle assembly		
2-1	202934-72502	1	Bottom Steering Stem	779XLS	
2-2	201401-72500	1	Bearing	779XLS	
2-3	202934-72501	1	Upper Steering platform	779XLS	
2-4	101501-72500	1	Front Fork	779XLS	
2-5	102404-72500	1	Brake	779XLS	
2-A	105101-72500	1	Front Wheel Assembly	779XLS	
2-6	103514-72500-50	1	Front Fender	779XLS, Black	
2-7	202934-72504	1	Front Fork Axle	779XLS	
			-----*-----		
2-B	101102-72500	1	Tiller w/ Revo-shift Lever	779XLS	
2-8	101101-72500	1	Tiller Frame	779XLS	
2-9	208203-72500	1	Button Switch	Left	
2-10	208203-72501	1	Button Switch	Right	
2-11	201102-72500	1	Throttle Handle	H-378-24V-SC	
2-12	201103-88100	2	Hand Grip	Black	
			-----*-----		
2-13	203504-72500	1	Charger inlet cover	779XLS	
2-14	207006-88108	1	Charging cable	KA-888-H10	
2-15	202934-72503	1	Block	779XLS	




[back to contents](#)

Index NO.	Part NO.	Qty.	Description	Specification	Remark
3-A	800317-725-00	1	Deluxe Seat Assembly	Black Vinyl, w/ Slider & Reclining	
3-1	104017-00328	1	Captain Seat	Black, Vinyl, Reclining	
3-2	204901-88803-A3	1	Sliding Rail	Left, #5	
3-3	204902-88803-A3	1	Sliding Rail	Right, #5	
3-4	5003-08020-221	4	Button Head Allen Bolt	M8*P1.25*20L,Multicolored Galvanized	
3-5	5021-08022-211	4	Flat Washer	Dia 8*22*5t,Multicolored Galvanized	
3-6	204506-67200-A2	1	Seat Adapter	889XL SM, A2	
3-7	204507-66103	1	Swivel Seat Stand	#4	
3-8	4001428-00	4	Flat Washer	A428, \$ 8.4*26*3t	
3-9	5012-08013-200	4	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	
3-10	204508-88805-A4	1	Seat Post Adaptor	165L, A4	
3-11	5008-10020-212-A2	2	Tighten Knob	M10*17L,Multicolored Galvanized,A2	
3-12	209931-00105	2	End Cap	30MM*30MM,2T	
3-13	5002-10016-720	2	Cap Head Allen Bolt, 12.9 Grade	M10*P1.25*16L,Black Galvanized	
3-14	5001-10016-710	1	Hex Head Screw, Nyloc	M10*P1.25*16L, Black Galvanized	
-----	301174-31	1	Caution Sticker	N74-31	
			----- * -----		
3-15	202922-88205	1	Connecting pin	787N	
3-16	5002-10016-720	1	Cap Head Allen Bolt, 12.9 Grade	M10*P1.25*16L,Black Galvanized	

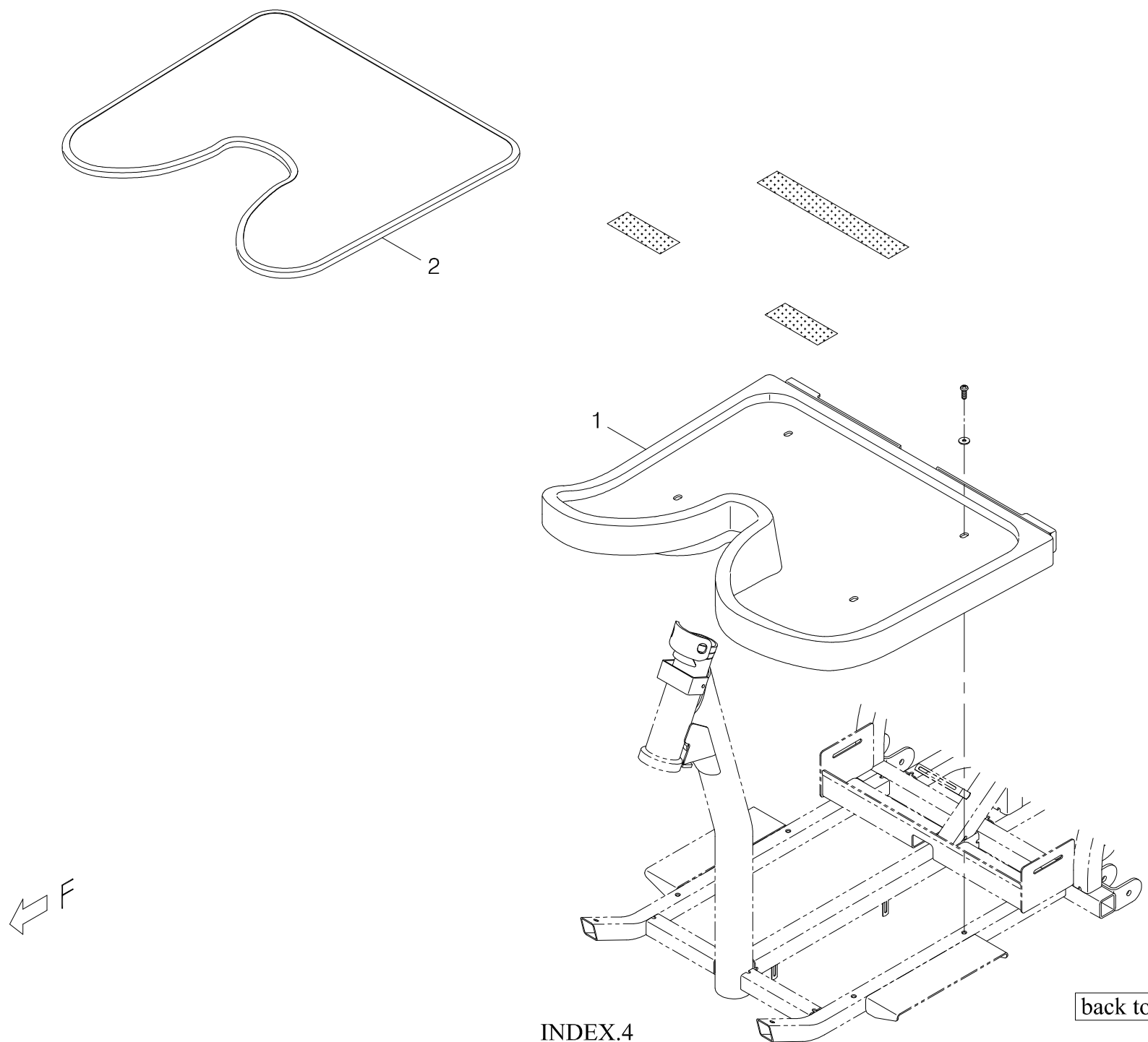

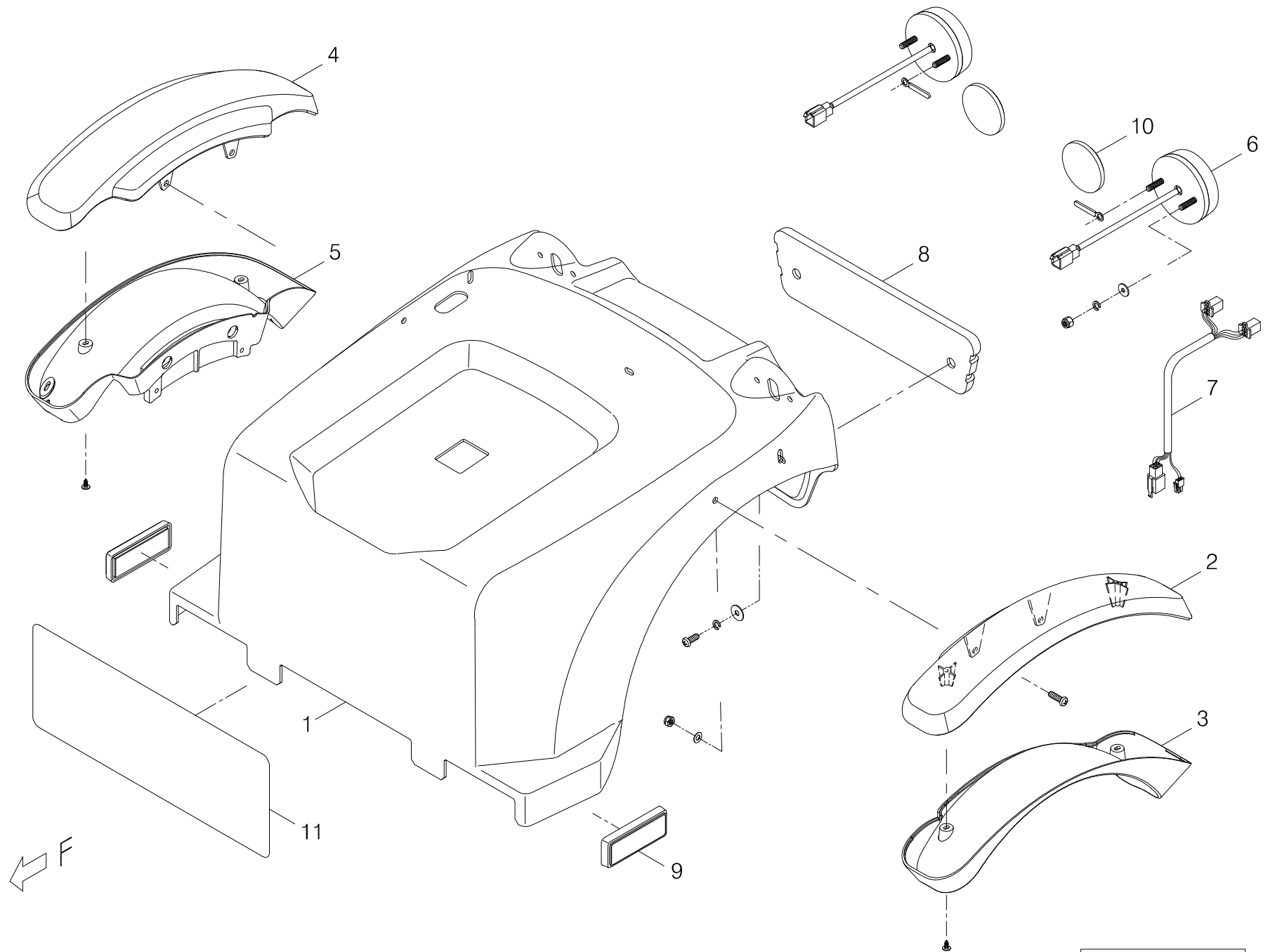

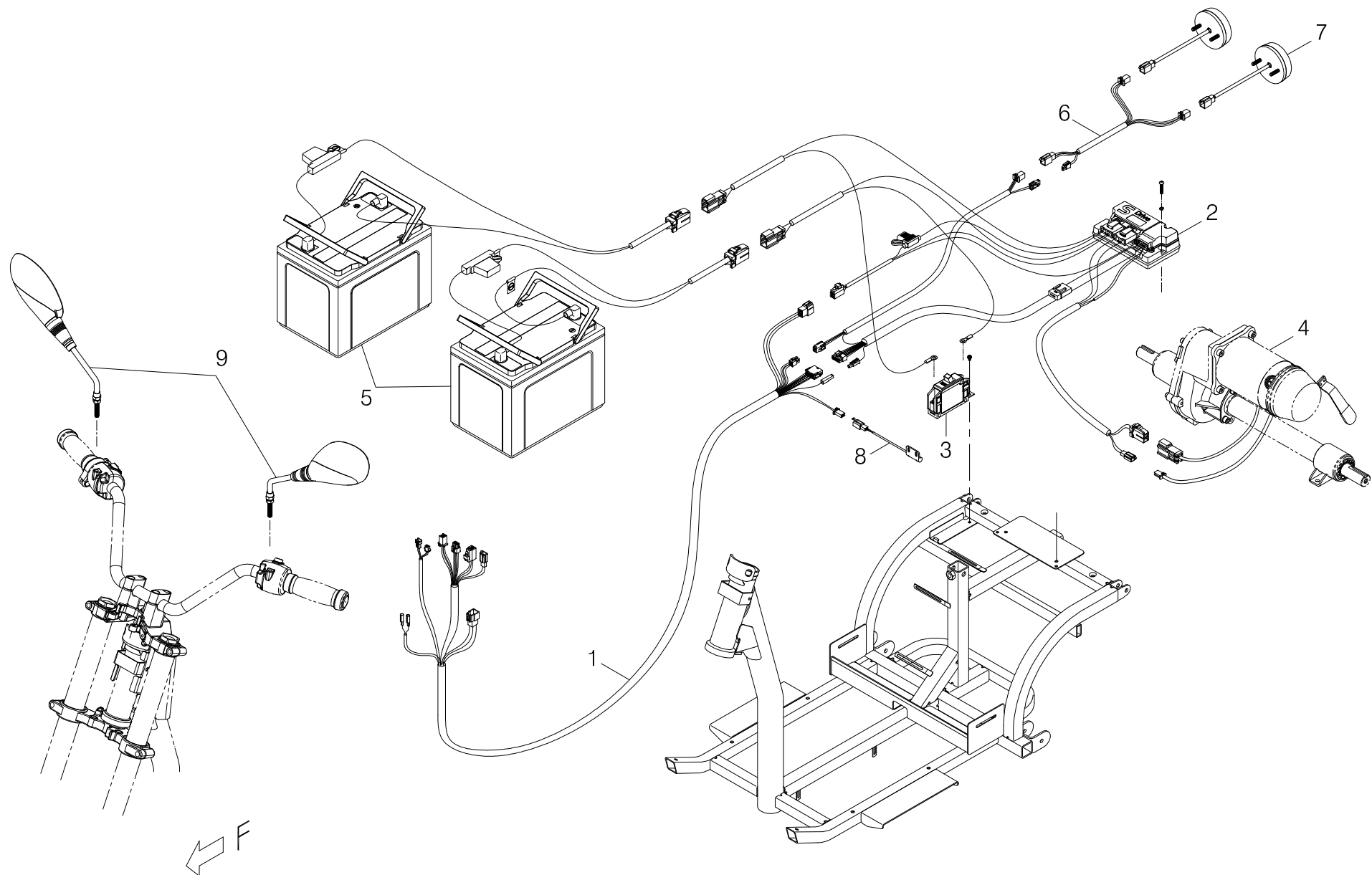


All Stickers and Decals shown as below and used on TE-779XLS

SHOPRIDER®

209939-68500



301116-01

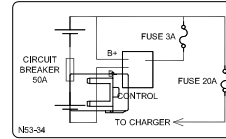


301153-34

EMERGENCY BRAKE


301165-00



301166-02

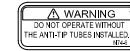


301174-06

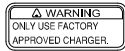

301174-16



301174-31

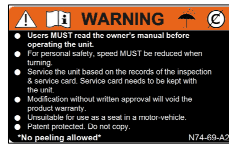


301174-69-A2



301174-74




301179-04

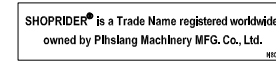

301180-96



301180-103

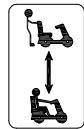

301187-13