
# rollz®

## Rollz Motion Electric

**EN - User manual**  
**NL - Gebruikshandleiding**  
**DE - Bedienungsanleitung**  
**FR - Manuel d'utilisation**

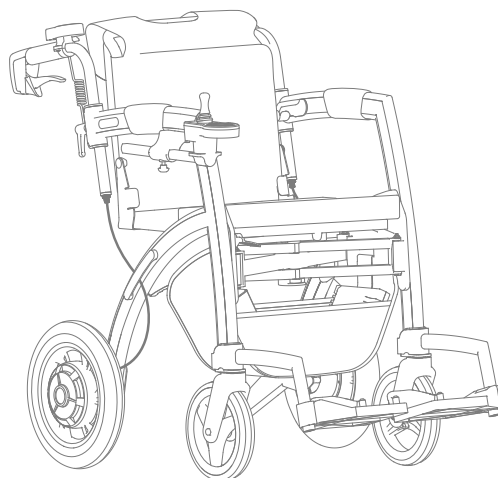**CE**

EN - User manual	5
NL - Gebruikshandleiding	61
DE - Bedienungsanleitung	119
FR - Manuel d'utilisation	177

For manuals in other languages,  
please check **[www.rollz.com](http://www.rollz.com)**.

Contact details of distributors per country. Check [www.rollz.com](http://www.rollz.com) for the complete list.

Belgium & France  
Mobio  
003293287723  
[www.mobio.be](http://www.mobio.be) & [www.mobio.fr](http://www.mobio.fr)

Germany & Austria  
Saljol  
+49(0)8141317740  
[www.saljol.de](http://www.saljol.de)

Switzerland  
PROMEFA AG  
+41448729779  
[www.promefa.ch](http://www.promefa.ch)

The Netherlands  
Rollz International BV  
+31 20 362 2010  
[www.rollz.nl](http://www.rollz.nl)

## USER MANUAL

The rollator and electric wheelchair in one

rollz motion electric

EN

## Rollz® International BV – Rollz Motion Electric

© Copyright 2023 Rollz® International BV

This is the original user manual of the Rollz Motion Electric. This user manual is based on the information available at the time it was created. It is possible that the information in this document will be changed. The purchased product can be slightly different to the one described in this user manual.

You can download the most recent manual at [www.rollz.com](http://www.rollz.com). To keep you updated on the newest information, we revise [www.rollz.com](http://www.rollz.com) regularly.

Rollz® International BV cannot be held responsible for technical errors or imperfections in this manual.

### CE conformity


The product fulfills the requirements of the EU-Directive 93/42 EEC for medical devices. Based on the classification rules set out in Attachment IX of this directive, the product is classified into class I. The declaration of conformity has been done under the own responsibility of Rollz® International BV based on the Attachment VII of this directive.

### International standards

The Rollz Motion Electric is certified as a rollator according to ISO standard 11199-2:2005. The Rollz Motion Electric is also certified as a wheelchair in accordance with ISO standard 7176-8: 2014 (Cor. 2015-09). The Rollz Motion Electric is certified as a class A electric wheelchair in accordance with ISO 12184:2014.

**rollz motion  
electric**

Max 100 kg / 220 lb

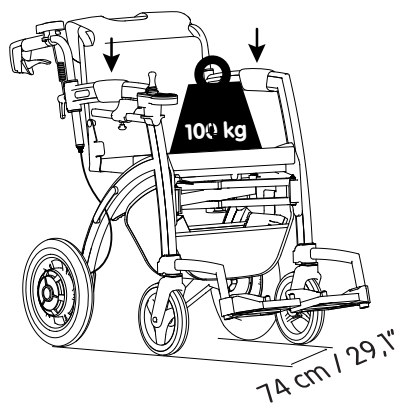**Figure 1: Rollz Motion Electric width and max load.**

## CONTENTS

<b>Before you start</b>	<b>10</b>
Important information	10
Safety instructions	11
Warnings	12
Unsuitable circumstances	13
Electromagnetic interference	14
Who may benefit from this product	14
Who should not use the Rollz Motion Electric (contraindication)	15
Reuse	15
<b>Unpacking</b>	<b>16</b>
Check if your delivery is complete	16
Attaching the push support	16
Attaching the joystick controller	17
Attaching the 3-in-1 holder	17
Charging the Rollz Motion Electric	18
Warnings	19
Recycling packaging material	19
<b>Use as rollator</b>	<b>20</b>
What fits where?	20
Unfolding & folding	21
Transport	22
Adjusting the height for correct posture	23
Walking with the Rollz Motion Electric - as a rollator	25
Operating the brakes	26
Resting on the Rollz Motion Electric	27
Using the bag	27
Going over thresholds and curbs	27
<b>From rollator to electric wheelchair</b>	<b>28</b>
What fits where on the Rollz Motion Electric	29
Attaching the Rollz wheelchair package	30
Using the Rollz wheelchair package	31
Setting the footrests (once only)	33
Setting the backrest (once only)	35

Putting down the wheelchair package	35
Removing the wheelchair package	36
Safety instructions	36
<b>Using as electric wheelchair - with a person pushing</b>	<b>37</b>
Matching your pace	37
Use of the electric support system - with a person pushing	37
Power assist modes	38
Constant power assist	38
Auto-power assist	38
Anti-fall	38
Going up a curb	39
Getting off the pavement safely	39
Slopes	40
<b>Use of the joystick- while sitting in the electric wheelchair</b>	<b>41</b>
Horn	41
Reversing	42
Going up a curb	42
Getting off the pavement safely	42
Slopes	43
Transfers (getting in or out of the wheelchair)	43
Seating position	43
<b>Specifications for Rollz Motion Electric</b>	<b>44</b>
Product specifications	44
Information about the material	44
Product measurements	45
Ergonomics dimensions	47
Technical details	47
<b>Error codes and solutions</b>	<b>48</b>
Error code listing	48
<b>Accessories</b>	<b>49</b>
<b>Maintenance</b>	<b>49</b>

Cleaning/disinfecting	49
Disconnecting the battery	50
Reconnecting the battery	51
Storing the Rollz Motion Electric	52
Battery maintenance (Daily)	52
Cables and connections (Weekly)	52
Motors (Every month)	53
Push support controller and joystick controller (Every 3 months)	53
Washing the seat cover	53
Brake performance (Daily)	54
Adjusting the brakes	54
Brake adjustments at the top of the brake cable	54
Brake adjustments at the bottom of the brake cable	55
Inflating the pneumatic tires (approximately every month)	56
Flat tires	57
Replacement of parts	57
<b>Warranty</b>	<b>57</b>
Warranty provisions	57
Register your Rollz Motion Electric	58
<b>Contact</b>	<b>59</b>

## BEFORE YOU START

Congratulations on your new Rollz Motion Electric. Rollz International BV has created this manual so you will use your rollator and electric wheelchair in one in the safest and most convenient way. If you are uncertain about the warnings or instructions mentioned in this manual, please contact your supplier for clarification.

We hope you will enjoy the use of your Rollz Motion Electric.

### Important information

- Read these directions carefully before you start using the product and keep them for future reference.
- This manual is written for the Rollz Motion Electric (introduced in 2022).
- Do not risk your safety and the safety of others around you by using the product incorrectly.
- In this manual we speak about the electric support system, this system contains the joystick controlled electrical support and the push support, which control the motors in the wheels.
- Colours of the Rollz Motion Electric parts may slightly fade when exposed to sunlight.
- Do not expose the product to temperatures below -5 °C / 23 °F and higher than 45 °C / 113 °F as this may cause damage to some components and affect their performance.
- The Rollz Motion Electric contains electronic components. For this reason, avoid getting the rollator wet. Do not use it outside in heavy rain.
- Only use the supplied charger to charge the Rollz Motion Electric.
- Use original Rollz accessories only. The use of non-approved accessories can be unsafe and will void the warranty.
- Use only original Rollz International BV approved parts for repairs or replacements made by an authorized dealer. Use of any other parts can be unsafe and will void the warranty.
- Damage due to overloading the product or incorrect folding and unfolding does not only void the warranty but also creates unsafe situations. You will find the warranty provisions in this manual.

- The serial and article numbers of your Rollz Motion Electric can be found on the product sticker, which is located on the seat (under the cushion).
- Any serious incident related to the device must be reported to the manufacturer and the competent authority of your country/state.
- It is important to register your product at Rollz International on [www.rollz.com](http://www.rollz.com).
- Please follow the rules that apply in your country regarding the use of electric wheelchairs.

### Safety instructions

	<p>Read and follow the instructions in this manual.</p>
	<p>Flammable materials: avoid contact with open flames or sparks. Prevent the battery from coming into contact with flammable materials.</p>

- The Rollz Motion Electric is meant to be used for supporting/transporting one person at a time.
- Always use the handles in the right position. Read the chapter **Adjusting the height for correct posture** for more details.
- Check and/or adjust the brakes regularly to prevent loss of braking power. Check the brakes regularly to make sure that you have adequate braking power. See how to do that in the chapter **Adjusting the brakes**.
- Check if all the power supplies are connected and the electrical components, including the battery, are not damaged before using the Rollz Motion Electric for the first time. Damaged or improperly connected parts may cause danger or unsafe situations. For more information about this, read the chapter **Cables and connections (Weekly)**.
- Joystick controlled electrical support is not necessarily capable of negotiating outdoor obstacles.
- While seated in the Rollz Motion Electric and using the joystick controlled electrical support always have the handlebars in rollator position, with the handlebars above the frame, so you can reach the handbrakes in case of emergency.
- Electrical support given by the push support controller might be used outside on surfaces that are considered safe in this manual.

- When seated in the wheelchair, while using the Rollz Motion Electric with joystick controlled electrical support, only go up or down (banked) slopes of max. 3° / 5.25%.
- When walking with the Rollz Motion Electric, only walk down on 15° / 26.8% slopes, walk up on 7° / 12.3% slopes and walk on banking slopes not higher than 3.5° / 6.1%.
- Check the capacity of the battery pack regularly by viewing the screen at the joystick or push support to see if it needs to be recharged.
- Keep tools and other metal objects away from the battery pack. Contact with the positive and negative poles can cause a short circuit or a shock.
- Make sure you are familiar with the functionalities of the Rollz Motion Electric before using it.
- Make sure that the Rollz Motion Electric is turned off when not in use.
- Store the Rollz Motion Electric in a clean and dry environment and avoid exposing it to high and rapidly changing temperatures. When not in use for a longer period of time, make sure the battery is charged before using it again.
- This product should be kept away from acid, alkali and other chemical corrosive items.

## Warnings

- The Rollz Motion Electric is certified as a Class A wheelchair, which means that outdoor use while seated in the Rollz Motion Electric with joystick controlled electrical support is limited to: maximum safe slope of 3° / 5.25%, maximum obstacle height of 15 mm / 0.6 inch and maximum ground unevenness of 10 mm / 0.4 inch. Exceeding these limits can lead to dangerous situations. Safe outdoor use can only be guaranteed in areas where these conditions are respected.
- It is essential the user has the physical and mental ability to independently perform all the operations in this manual. If not, they should always have someone to assist them.
- Do not use the Rollz Motion Electric if any part has been damaged, broken, missing or cracked, or if the displays show an error code, for solutions of these error codes, see chapter ***Error codes and solutions***.

- Make sure all components are locked in their place before use.
- Keep the Rollz Motion Electric away from open flames and other heat sources, such as cigarettes and electrical or gas heaters.
- The user cannot remain seated in the Rollz Motion Electric wheelchair during transport in a taxi, bus, car or other motor vehicles. It is not meant for this type of usage.
- The width of the wheelchair is greater than what is recommended for Class A wheelchairs. Access to emergency escape routes might be limited.
- When using the joystick controlled electrical support or the push support of the Rollz Motion Electric, the continuous driving range is 15 kilometres / 9.3 miles. The driving range will decrease when the Rollz Motion Electric is used on slopes, rough terrain or when used to climb kerbs. User weight, terrain, battery status and driving behaviour can also affect the range.
- The Rollz Motion Electric was not intended for use on the beach.
- We advise using the seatbelt accessory while you are seated in the Rollz Motion Electric.

### **Unsuitable circumstances**

- To board a plane, the battery on the Rollz Motion Electric must be removed. The battery can't be transported in a plane.
- The Rollz Motion Electric is not suitable for climbing stairs and walking on rough terrain such as gravel, grass, sand, mud or forest soil.
- Do not use the escalator while seated on your Rollz Motion Electric.
- Do not use or park your Rollz Motion Electric in the rain, snow or other wet conditions.
- Do not use your Rollz Motion Electric while showering.
- Do not use your Rollz Motion Electric on roads where salt is used.
- Do not use your Rollz Motion Electric too close to water (e.g. ditch, pond, river, lake or sea).
- Do not use the Rollz Motion Electric on a surface with snow, ice, water or oil.
- If you however do find yourself with the Rollz Motion Electric on a wet slippery surface, please proceed with caution and if you notice that one of the driving

wheels is losing traction, you should stop the Rollz Motion Electric immediately to prevent loss of control.

- Do not expose the Rollz Motion Electric to temperatures colder than -5 °C / 23 °F and higher than 45 °C / 113 °F.
- Surface temperatures of the product can increase / decrease when exposed to external sources of heat or cold.
- Not all parts of the Rollz Motion Electric are 100% waterproof. A very humid environment may cause rust and corrosion on or inside your Rollz Motion Electric. When you are in an excessively humid environment, you should immediately cover the Rollz Motion Electric to prevent damage.

### **Electromagnetic interference**

Interference with electronic sources can cause situations in which the Rollz Motion Electric does not respond as usual. Electrical sources such as broadcasting masts, radio stations, television stations and mobile phones can create a magnetic field that may cause permanent damage to the electric system and/or braking capabilities.

The user should observe the following:

- Do not use wireless communication devices while using the electric support system.
- Stay away from radio sources.
- Turn off the Rollz Motion Electric immediately if you notice uncontrolled movements.

The Rollz Motion Electric might interfere with other devices in the area that emit electromagnetic fields (alarm systems, automatic doors, etc.).

### **Who may benefit from this product**

The Rollz Motion Electric fits people weighing up to 100 kg / 220 lbs. / 15 st 10.5 lb, who are 160-190 cm / 5 ft 3" to 6 ft 3" tall (see **Product Specifications**).

The Rollz Motion Electric serves as walking support for people with reduced mobility, mainly indoors (joystick controlled and push attendant controlled) but also outdoors (push attendant controlled and joystick controlled, with limitations as mentioned in this manual). The Rollz Motion Electric is suitable for walking, driving and pushing on hard, flat surfaces. While in rollator position, the user can rest on the seat.

The wheelchair package offers the possibility to transform the rollator into an electric wheelchair easily. The user can sit in the electric wheelchair and use the electric support system, whilst being pushed and having the push support turned on, or by transporting themselves by using the joystick controller.

### **Who should not use the Rollz Motion Electric (contraindication)**

The Rollz Motion Electric is not suitable for people:

- With perception disorders which make it difficult to see obstacles ahead.
- With severe balance disorders that prevent them to walk with a rollator.
- Who get disorientated or dizzy while walking or sitting.
- Who can only remain standing for less than five minutes behind a walker;
- With seating problems, which make it difficult to sit comfortable on the seat or in the electric wheelchair.
- People who have an impaired hand function.
- With a weight higher than 100 kg / 220 lbs.

### **Reuse**


- The Rollz Motion Electric is suitable for reuse.
- For safe reuse, pass all technical documentation on to the new user. The newest version of the manual can be found on [www.rollz.com](http://www.rollz.com).
- When lending your Rollz Motion Electric, make sure the user reads the manual before using the product.
- Before passing the Rollz Motion Electric on to a new user, it should be cleaned, disinfected and checked by the dealer for damage to make sure the Rollz Motion Electric can be safely used.

## UNPACKING

### Check if your delivery is complete

The Rollz Motion Electric comes packed. Check if all the parts listed below have been supplied according to **(Figure 2)**:

1. Push support controller
2. Wheelchair package
3. Armrests (x2)
4. Seat cushion
5. Motor controller
6. Adjustable footrests (x2) with Allen key
7. Rear wheel motor assembly (x2)
8. Li-ion battery pack
9. Joystick controller
10. Adjustment handles
11. Underseat basket
12. 3-in-1 holder
13. Pump for the pneumatic tires

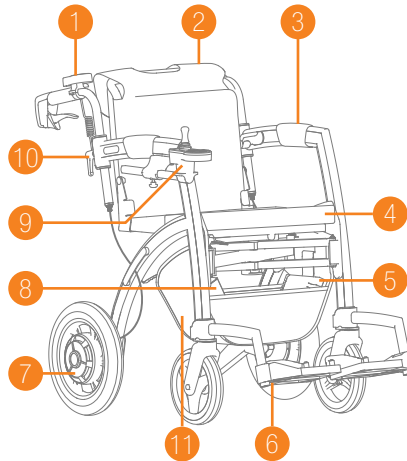
**Figure 2: Overview of the components of the Rollz Motion Electric.**

### Attaching the push support and joystick controller

Before you can use the Rollz Motion Electric, the two controllers need to be attached. Without properly attached controllers, the Rollz Motion Electric might behave unexpectedly and in an unsafe manner.

#### Attaching the push support


- a. Attach the push support controller **(Figure 3)**. A click can be heard when the correct position is reached.

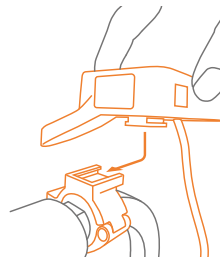
**Figure 3: Attach the push support controller.**

## Attaching the joystick controller

- a. Make sure the cable of the joystick controller is disconnected before attaching it. Disconnect by loosening the metal rings. Slide the disconnected joystick controller through the hole in the holder. Make sure that the cable runs between the frame and the controller as shown in **(Figure 4)**.
- b. Reconnect the cable, and tighten the controller by rotating the knob **(Figure 5)**.

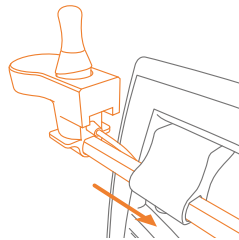**Figure 4:** Slide the joystick controller into the holder.

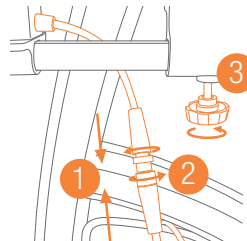**Figure 5:** Connect (1&2) and tighten (3) the controller.

## Attaching the 3-in-1 holder


The 3-in-1 holder can be put in place without any tools:

- a. Slide the armrests as far back as possible, the two holder should be placed in front of the armrests, with the shopping hooks facing outside, as seen on **Figure 6**.
- b. Attach the holders by pressing it down onto the frame **(Figure 6)**.
- c. The 3-in-1 holder can hold the wheelchair package (above the seat), a cane and a shopping bag at the same time.
- d. Follow the instructions on the manual that comes with the 3-in-1 holder to see how to place the wheelchair package, the cane and the bags into the holders.
- e. The Rollz Motion Electric can also be used without the 3-in-1 holder.


### NOTE

Do not place more than 2.5 kg / 5.5 lb in each bag and do not overload it.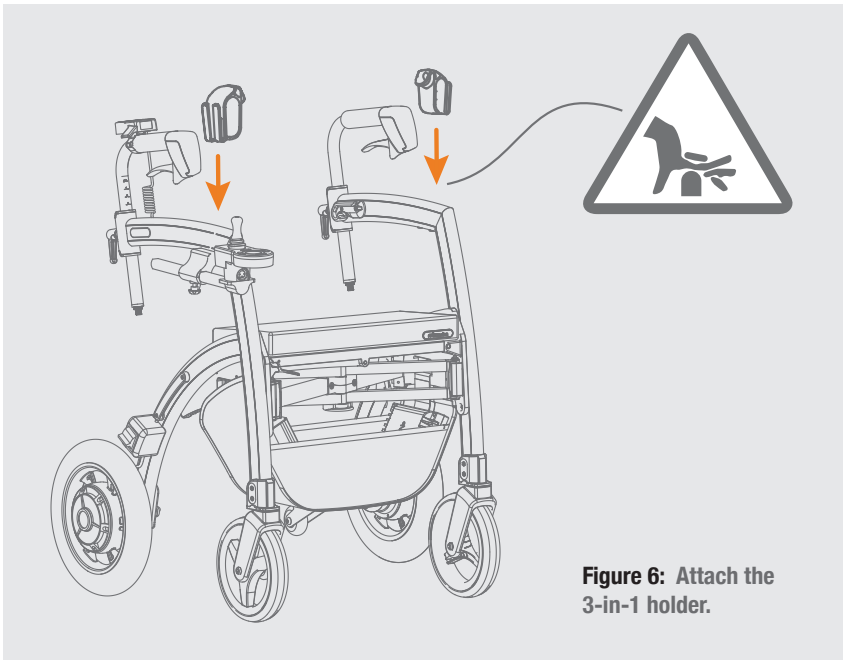

### Charging the Rollz Motion Electric

The battery is charged via the charging port under the joystick controller (**Figure 7**). Plug the connector of the delivered charger into the charging port. Plug the charger into a wall socket. The Rollz Motion Electric is charging when the indicator light on the charging adapter turns red. The battery is completely charged when the indicator light turns green. Charging will take 4-5 hours.

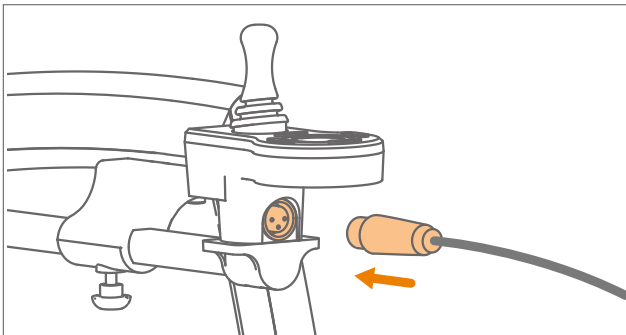**Figure 7: Plug the connector into the charging port.**

## Warnings

- Do not use metal objects or tools to connect the battery to the charger.
- When the battery is being charged, do not move the Rollz Motion Electric.
- Do not use unapproved parts on the Rollz Motion Electric (such as motor and charger).
- Do not pinch or twist the charger cord.
- Smoking and flames near a charging battery are prohibited.
- Keep the charger away from children and pets.
- When a fault occurs in the electronic circuit, inform your supplier as soon as possible.
- Place the battery and charger in an open space during charging. Put no objects on the charger, battery or cord. Do not place near heat sources during charging.
- Do not use the battery and charger in temperatures below -5 °C / 23 °F and above 45 °C / 113 °F.
- Keep tools and other metal objects away from the battery. Contact with metal objects can cause shock and short circuit.
- Keep the battery away from flammable materials. Avoid exposure to open flames and sparks.
- There are corrosive substances in the battery. Never open the battery.

## Recycling Packaging Material

All the packaging material can be recycled. The cardboard boxes can be recycled with paper. The protective foam blocks are made of plastic. These and the plastic bags can be recycled with plastic.

*TIP:* Keeping your box will assist with sending your Rollz Motion Electric via post.

## WARNING

Keep the plastic packaging materials away from children due to the risk of suffocation.

## USE AS ROLLATOR

The Rollz Motion Electric is delivered complete for use. You only have to attach the push support and joystick controller (see **Attaching the push support and joystick controller**) and adjust the handles to the correct height. Please note that you shouldn't use the Rollz Motion Electric as a rollator with the electric support turned on.

Read all points carefully before using the Rollz Motion Electric rollator:

- What fits where?
- Unfolding & folding
- Transport
- Adjusting the height for the correct posture
- Walking with the Rollz Motion Electric - as a rollator
- Operating the brakes
- Resting on the Rollz Motion Electric
- Using the bag
- Going over thresholds and curbs


### What fits where?

**(Figure 8):**


1. Handles
2. Frame
3. Rear wheels with drum brakes
4. Red release handle for folding up the rollator
5. Front wheels
6. Front

**(Figure 9):**

7. Handbrakes
8. Parking brake levers
9. Height indexation
10. Adjustment handles
11. Adjusting screws for adjusting the brakes
12. Tipping aid
13. Seat cushion

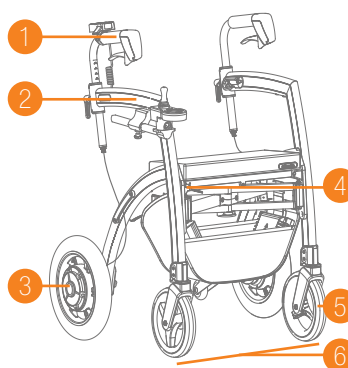**Figure 8: What fits where.**

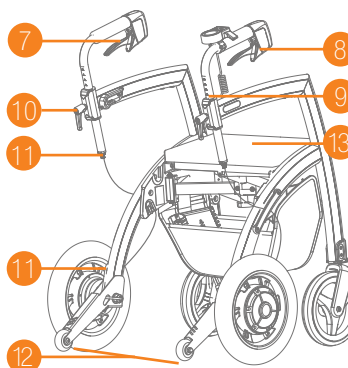
**Figure 9: What fits where.**

## Unfolding & folding


When folded, the Rollz Motion Electric fits in most cars.

Unfolding goes as follows:


- a. Pull the frame sides of the Rollz Motion Electric apart until almost completely unfolded (**Figure 10**).
- b. Push the seat down until it clicks into place. Important: the seat is not secured until you have clearly heard it 'click' (**Figure 11**).
- c. Pull away the seat cushion and place it straight on the seat (**Figure 12**).

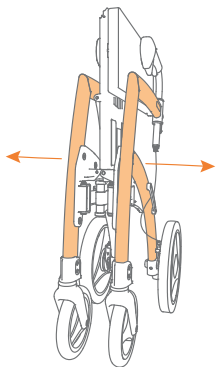**Figure 10:** Unfold by pulling the frame outwards.

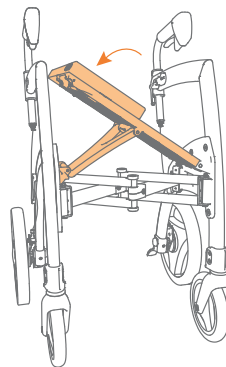**Figure 11:** Push the seat down till you hear a 'click'.

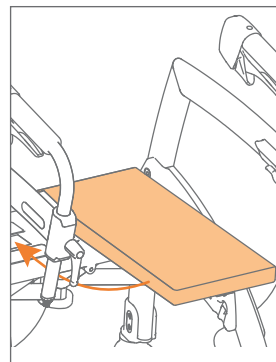**Figure 12:** Pull away the cushion and attach it on the seat.

### CAUTION

Double check if the seat is locked in. Carefully lift the seat in the middle to make sure that it is properly locked in place. Only then the Rollz Motion Electric can be safely used.

Folding goes as follows:

- a. Before you fold the frame, the underseat basket should be emptied, cables, battery pack and motor controller must stay in the underseat basket.
- b. Remove the wheelchair package if it is placed on the rollator. Read **Removing the wheelchair package** for instructions.
- c. Pull the seat cushion away, turn it 90 degrees and place it crosswise on the side of the red handle (**Figure 13**).


- d. Push the red release handle under the seat (**Figure 14**) and lift the seat partially (**Figure 15**).
- e. Push the sides of the frame towards each other until the rollator is completely folded (**Figure 16**).

**NOTE**


When you fold the frame, the underseat basket should only contain the cables, battery pack and motor controller, and the wheelchair package should be detached.

**Transport**


It's mandatory to switch off your Rollz Motion Electric during transport and it is strongly recommended that you apply the parking brake.

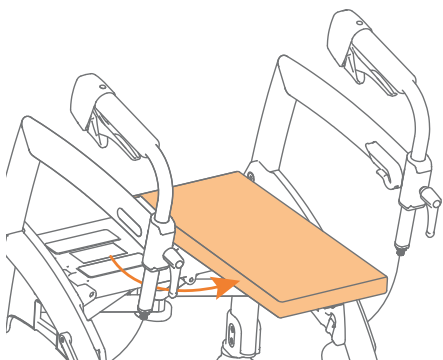
**Figure 13:** Place the seat cushion crosswise.

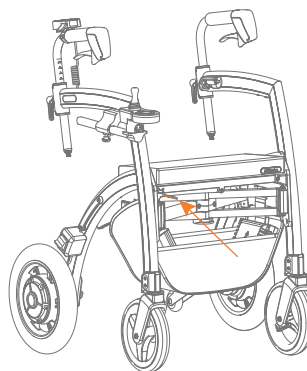**Figure 14:** Push the red handle while lifting the seat.

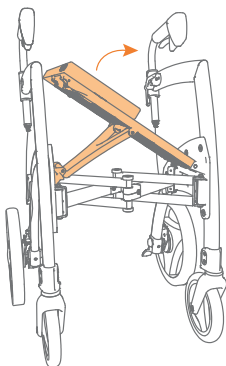
**Figure 15:** Lift the seat partially.

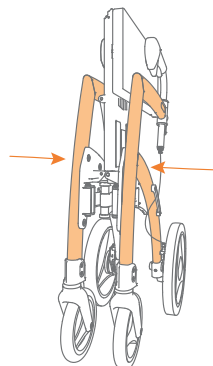**Figure 16:** Push the frame further inside.

During transport, make sure that your Rollz Motion Electric is placed in the designated storage space, such as the trunk. Make sure that the Rollz Motion Electric rests securely during transport and cannot move freely. This prevents any damage to you or your Rollz Motion Electric. Also be sure to disconnect and transport the joystick and push controller separately.

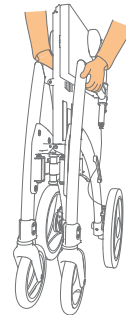
**Figure 17: How to lift the Rollz Motion Electric.**

Only lift the Rollz Motion Electric by frame parts as shown in **Figure 17**.


### Adjusting the height for correct posture

Setting the correct height (**Figure 18**) is crucial for an optimal use of the Rollz Motion Electric. The height can be adjusted in between the minimum and maximum positions. This way the Rollz Motion Electric can easily be adapted to individual needs.

- a. To determine the correct height, stand between the Rollz Motion Electric handles with your arms hanging near the body. The handles should be at the height of your wrists (**Figure 18**).
- b. Loosen the adjustment handles anticlockwise (**Figure 19**) and (**Figure 20**).
- c. Slide the handles to the correct height (**Figure 21**).

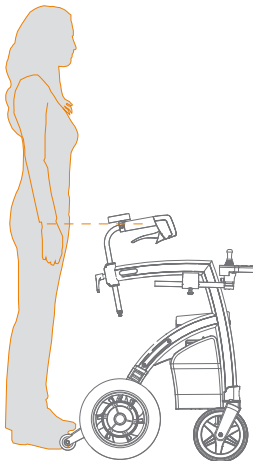
**Figure 18: Determine the correct height.**

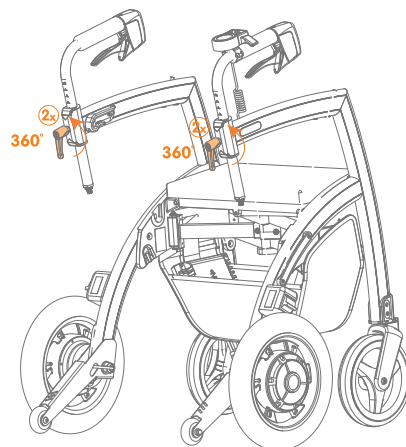
**Figure 19: Loosen the adjustment handles.**


**Figure 20: Rotate anticlockwise.**

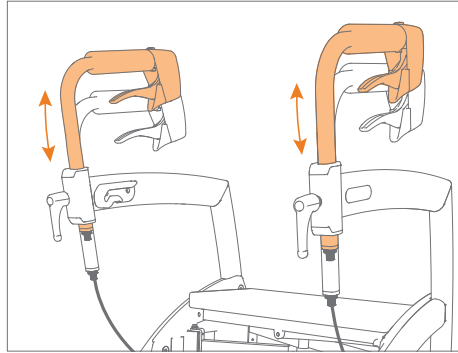
**Figure 21: Set the correct height.**

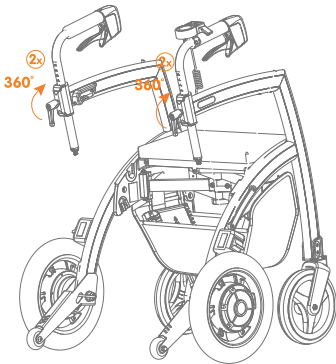
**Figure 22: Tighten the adjustment handles.**

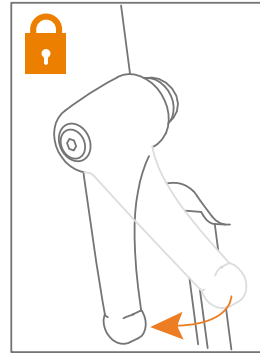**Figure 23: Tighten.**

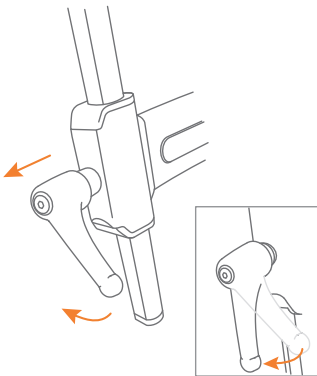**Figure 24: Set the adjustment handle in the right position.**

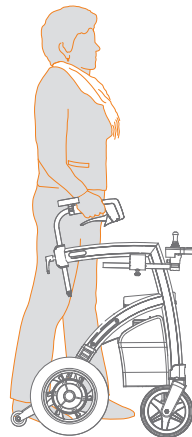**Figure 25: Walk upright close to the rollator.**

- d. Check whether both handles are at the same height using the height indexation.
- e. Make sure the handles are positioned in a straight line above the frame of the Rollz Motion Electric, pointing forward.
- f. Turn the adjustment handles clockwise to tighten them (**Figure 22**) and (**Figure 23**).
- g. Place the adjustment handles vertically so that they point straight down. You do this by gently pulling the adjustment handle outwards and turning it to the correct position (**Figure 24**).

#### CAUTION

Always set both handles at the same height to maintain stability. When height is incorrectly set, this may cause imbalance that may increase the risk of injury. When the rollator is not adjusted to the correct height, it can lead to neck or back pain.

### Walking with the Rollz Motion Electric - as a rollator

Thanks to the ergonomic design, you can walk upright, stable and comfortable with the Rollz Motion Electric.

- a. First make sure the handles are adjusted to the correct height. See above in **Adjusting the height for correct posture**.
- b. Stand in between the handles while walking. This way you walk as close as possible to the walker, which offers the most support (**Figure 25**). You could lean on the Rollz Motion Electric if necessary.
- c. Try to keep your arms and hands as relaxed as possible, this prevents too much tension on body parts.
- d. Keep an eye on your surroundings. Do not focus your eyes downwards, but forwards, which also improves the balance.

#### CAUTION

While you walk with the rollator, keep looking in front of you to determine a safe route.

#### WARNING

Prevent yourself from driving one or all the wheels into a hole or uneven terrain. The rollator could tilt diagonally and you might fall. The rollator could get damaged as well.

## Operating the brakes

The Rollz Motion Electric has advanced drum brakes that work in all weather conditions. The brakes are easy to operate and can provide varied braking power. The harder you squeeze the brake handles, the more you brake. The brakes can be locked when you park the Rollz Motion Electric so that it does not roll away. Test the braking power regularly, especially before the first use, to experience how they work.

- a. While walking, keep the fingers loosely around the foam handgrips (**Figure 26**). You can also keep them around the brake handles, with your thumb on the inside of the handle.
- b. Squeeze the brake handles for braking (**Figure 27**).
- c. Do you want to use the parking brake to make sure the rollator does not roll away? First squeeze the brake handles and simultaneously squeeze the parking brake levers. Then release the brakes. Your walker is now in the parking position (**Figure 28**).
- d. To remove the walker from the parking position, squeeze the brake handles again and release them (**Figure 29**). The rollator is not in parking position anymore.

### WARNING

When you walk and you squeeze the brake handles too hard, the Rollz Motion Electric can stop too quickly.

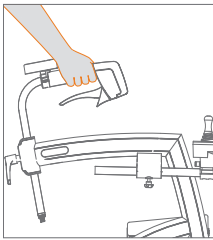
Figure 26: Driving.

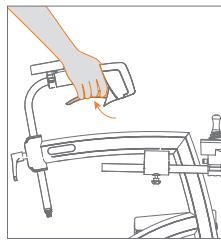
Figure 27: Braking.

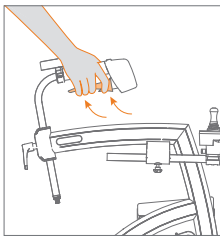
Figure 28: Apply the parking brake.

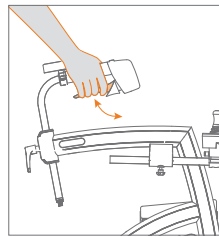
Figure 29: Release the parking brake.

## Resting on the Rollz Motion Electric

The Rollz Motion Electric rollator gives support when walking. If you want to take a break, you can also sit on it.

- a. Always apply the parking brake before attempting to sit (see **Operating the brakes**), otherwise the Rollz Motion Electric can roll away while attempting to sit on it or while sitting on it.
- b. Sit down on the rollator seat from the rear (**Figure 30**). Always ensure that the Rollz Motion Electric walker is in the parking position.


### CAUTION

Only sit and move yourself around on the Rollz Motion Electric when the wheelchair package is attached. If the wheelchair package is not attached, the walker is not suitable for moving yourself around whilst sitting. It is not designed for this purpose. You could fall while doing this, with all the associated risks.

## Using the bag

- a. You can reach into the bag under the seat from the rear of the Rollz Motion Electric.
- b. The bag can carry a maximum weight of 6 kg / 13 lbs.
- c. When the bag is empty, except for the cables, battery pack and motor controller, it can remain attached when folding the frame.

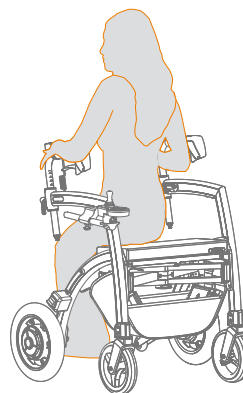**Figure 30:** Sitting on the rollator seat.

### NOTE

Do not overload the bag.


## Going over thresholds and curbs

To easily get on pavements up to a height of around 12 cm / 4.7 inch, or over thresholds, the Rollz Motion Electric has two tipping aids.


- a. Place the Rollz Motion Electric rollator with the front wheels against the threshold or curb.
- b. Push the left or right tipping aids (**Figure 31**) down with your left or right foot, while tilting the Rollz Motion Electric towards you with your hands. This way, the front wheels lift off the ground. Now push the Rollz Motion

Electric forward until you are clearly on the curb or over the threshold with the front wheels (**Figure 32**).

- c. Place the front wheels onto the sidewalk or ground.
- d. Then push the walker further forward until the rear wheels are also completely on the pavement or over the threshold.

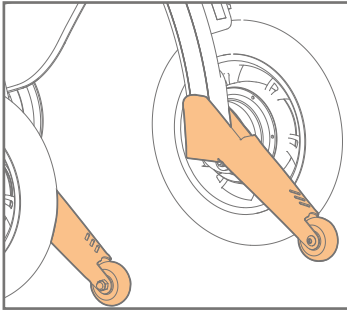**Figure 31:** Use the anti tipping aid.

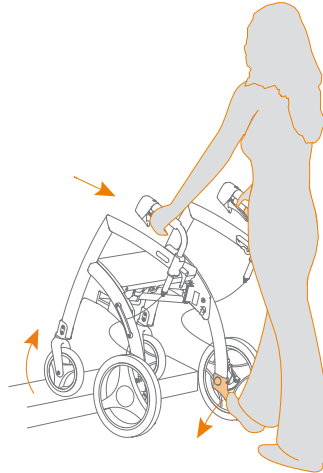**Figure 32:** Getting on the curb or pavement or go over a threshold.

## FROM ROLLATOR TO ELECTRIC WHEELCHAIR

The Rollz Motion Electric is delivered with the wheelchair package included. This package can be attached to the Rollz Motion Electric before departure (**Figure 33**). This allows the rollator to be transformed into an electric wheelchair on the road, in less than a minute. You only need to pull up the wheelchair package, place the footrests on the frame and turn the handles. Then you can sit in the electric wheelchair. With the electric support system you can also sit in the chair and drive yourself by using the joystick, or the person pushing the electric wheelchair can use the push support controller. The electric wheelchair can be easily turned into a walker again.

**CAUTION**

The Rollz Motion Electric wheelchair can carry a maximum weight of 100 kg or 220 lbs. This electric wheelchair is intended to be used by only one person at a time. When multiple persons are sitting on the electric wheelchair or rollator, the Rollz Motion Electric could be damaged.


**WARNING**


During transport in a wheelchair accessible taxi, bus, train or other vehicle, the user cannot remain seated in the Rollz Motion Electric. The electric wheelchair is not intended for this use. The Rollz Motion Electric can't be transported by plane, unless the battery is removed.

**What fits where on the Rollz Motion Electric**


1. Wheelchair package
2. Red handle
3. Locking bolts
4. Notch behind the seat
5. Upper guide rails on the frame
6. Backrest
7. Footrest holders
8. Footrests
9. Armrests
10. Handle wheelchair package

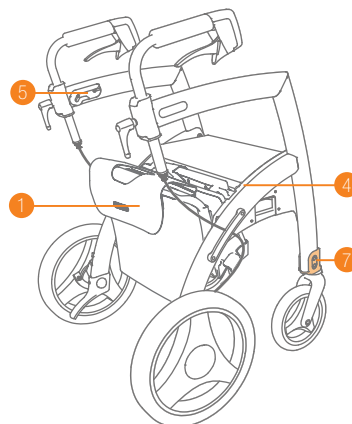**Figure 35: Rollz Motion Electric rollator with attached wheelchair package.**

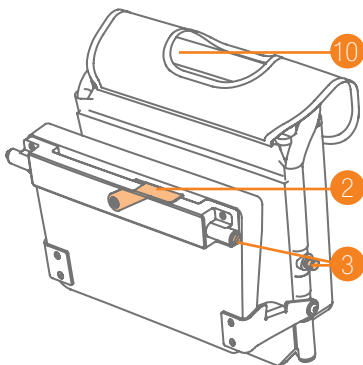**Figure 33: Wheelchair package.**

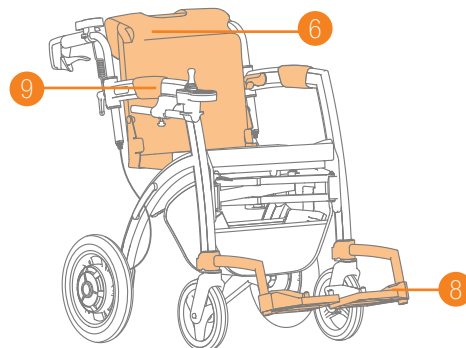
**Figure 34: Rollz Motion Electric in electric wheelchair with person pushing position.**

## Attaching the Rollz wheelchair package


- a. Hold the Rollz wheelchair package with the bag towards you (**Figure 36**).
- b. Lift the red handle with your other hand so that the locking bolts retract.
- c. Place the Rollz wheelchair package in the two notches behind the seat (**Figure 37**) and (**Figure 38**).
- d. Push the red handle back down without force to lock the wheelchair package. The locking bolts will extend so that the package is properly secured (**Figure 39**). You can now walk with the Rollz Motion Electric as a walker, carrying along the wheelchair package.

### NOTE


Forcing down the red handle can damage the Rollz Motion Electric.

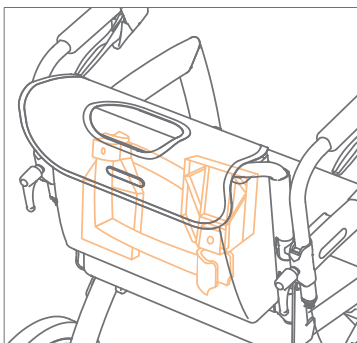
**Figure 36:** The footrests in the bag of the transport chair package.

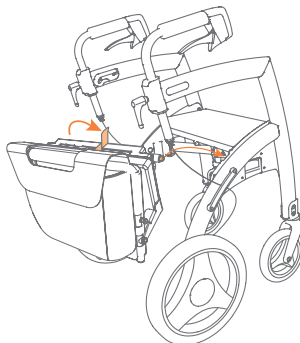**Figure 37:** Attaching the transport chair package.

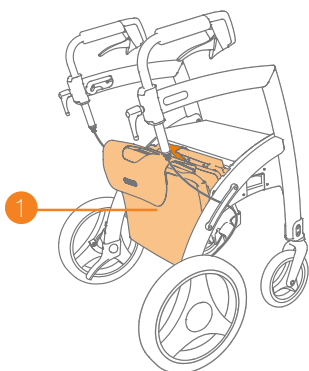**Figure 38:** Wheelchair package attached.

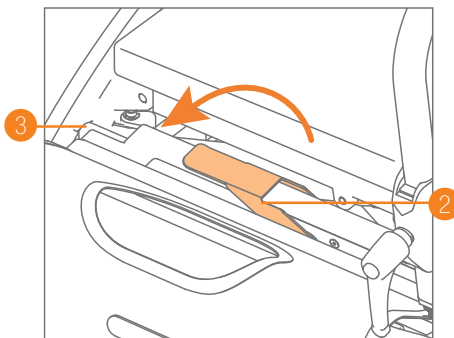**Figure 39:** Locking the wheelchair package.

- e. If the red handle is not going down without forcing it, take the wheelchair package out of the notches behind the seat and try again. Please contact your supplier if you still have difficulties after several attempts.

### Using the Rollz wheelchair package

- a. Attach the wheelchair package to the Rollz Motion Electric before you go out **(Figure 37)**.
- b. To convert the walker to an electric wheelchair, the wheelchair package needs to be pulled up. Take the handle of the wheelchair package and pull slightly up and towards the front until halfway up the frame **(Figure 40)**.
- c. Pull the backrest further up and then back **(Figure 41)** so that the locking bolts click into the upper guide rails on the frame **(Figure 42)**. If the upper locking pins do not slide smoothly into the upper guide rails on the frame, push the backrest forward until halfway up the frame and try again.

#### NOTE


If the wheelchair package does not pull up smoothly, do not apply force. This can damage the wheelchair package. Contact your supplier if the problem remains after several attempts.

- d. Loosen the adjustment handles anticlockwise **(Figure 43)** and turn the handles around via the outside, until they are pointing towards the back **(Figure 44)**. Adjust the height for the person pushing the electric wheelchair and turn the adjustment handles clockwise to tighten **(Figure 45)**. See **Adjusting the height for correct posture**.
- e. Take the footrests out of the wheelchair bag and place them at the front of the Rollz Motion Electric, into the footrest holders on the frame. Slide them down from the top of the footrest holders **(Figure 46)**. Do not apply force if the footrests do not slide smoothly over the footrest holders. Take them out and try again.
- f. The electric wheelchair is now ready for use **(Figure 47)**.


#### CAUTION


Before you sit down, always check that the locking bolts are correctly secured in the guide rails. Do not sit down if one of the locking bolts is not secured. Instead, try to place the seat again.

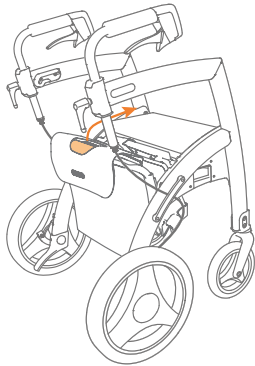
**Figure 40:** Pull the backrest upwards and forwards.

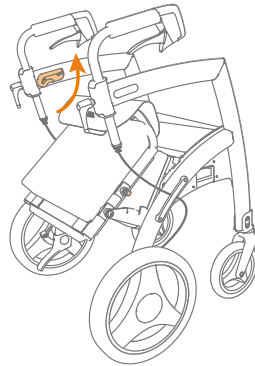
**Figure 41:** Pull the backrest further up.

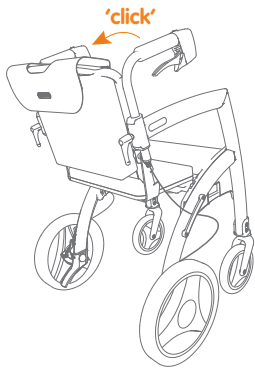**Figure 42:** Click the backrest into the guide rails.

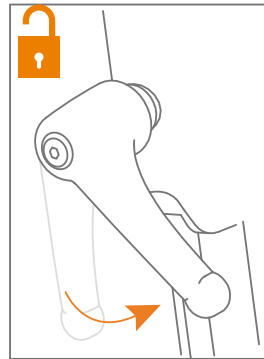
**Figure 43:** Loosen the adjustment handles.

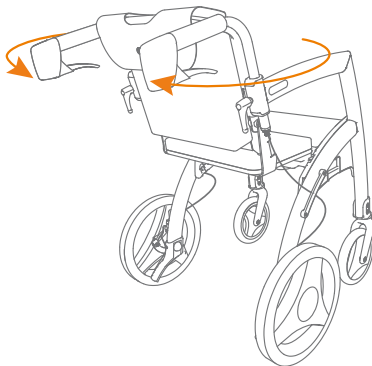
**Figure 44:** Turn handles.

**Figure 45:** Tighten the adjustment handles.

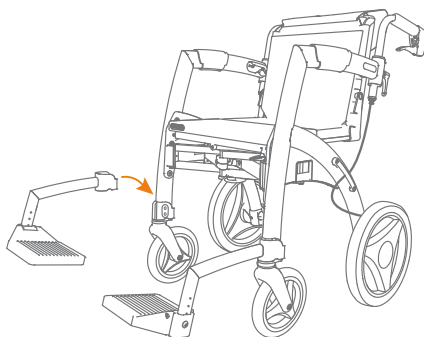**Figure 46:** Place the footrests.

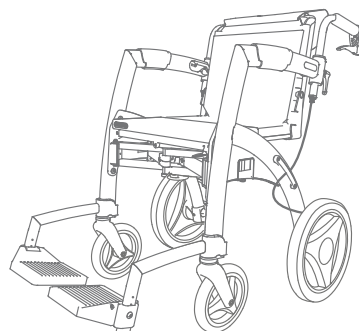
**Figure 47:** Electric wheelchair is ready to use.


### Setting the footrests (once only)

The Rollz Motion Electric is supplied with two adjustable footrests. You can adjust them to the desired height with the supplied size 5 Allen key (**Figure 48**).


The correct height depends on the length of the lower leg and the way the user sits. If the footrests are set too high, there will be too much pressure on the buttocks. If the footrests are set too low, the pressure on the upper legs is too high. In a correct position, the user sits with his lower legs at an angle of about 90 degrees (**Figure 49**). The back of the feet can rest against the fabric.

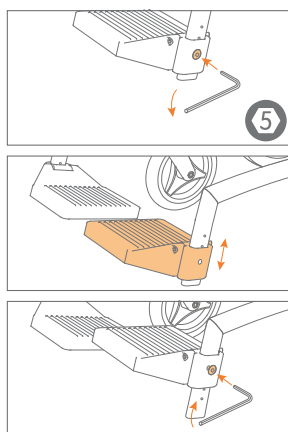
**Figure 48:** Adjusting the height of the footrests with an Allen key size 5.

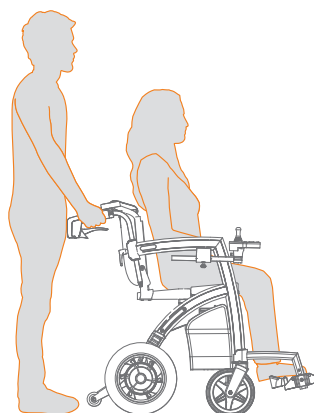
**Figure 49:** Good sitting position in the wheelchair.

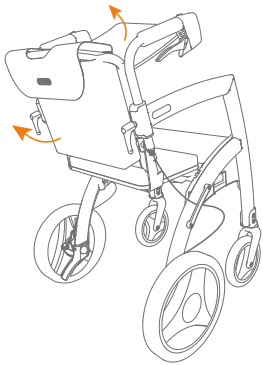
**Figure 50:** Clear adjustment straps.

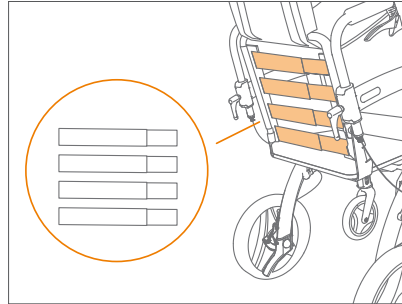
**Figure 51:** Four adjustable adjustment straps.

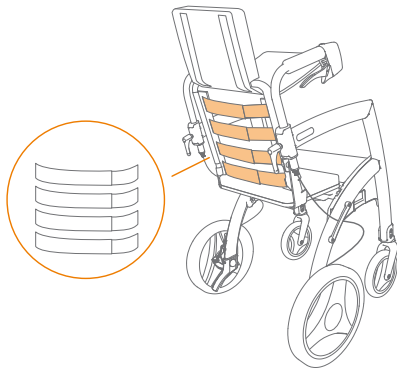
**Figure 52:** Loosen for more seat depth.

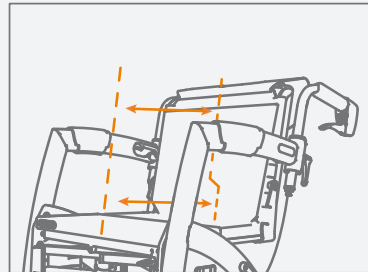**Figure 53:** Lower two back straps loosened.

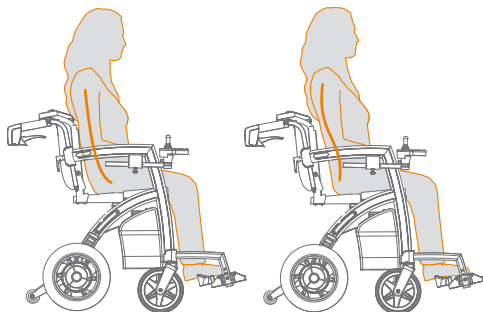**Figure 54:** Before and after adjusting the back straps, on the right you see the correct position.
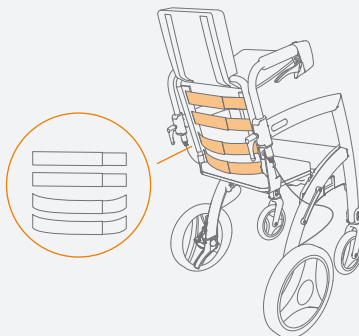
## Setting the backrest (once only)

Adjustment straps are placed in the backrest of the Rollz wheelchair package. For optimal seating comfort, it is important to adjust these adjustment straps to the user. It is only necessary to do this for the first use - or if the user has undergone major changes in weight.

- a. Turn the Rollz Motion Electric into an electric wheelchair (**Figure 47**) and remove the footrests from the wheelchair package bag.
- b. Pull up the bag of the wheelchair package and the backrest cover from the bottom (**Figure 50**). The adjustment straps are now clearly visible (**Figure 51**).
- c. Adjust the straps by pulling off the Velcro and fastening it again to the desired setting (**Figure 52**).
- d. In most cases, the lower straps will be adjusted looser than the upper two straps (**Figure 53**). Attach the Velcro so that the user's pelvis is properly upright, and the backrest supports the back.
- e. Put down the backrest cover and the wheelchair package bag.
- f. Check if the user is comfortably in a correct sitting position (**Figure 54**).

### CAUTION

Ensure the straps in the backrest are properly adjusted (see **Setting the backrest**). If the seat depth is large enough, the weight of the person is placed as far back in the seat as possible. That is safer for the user but also easier for the pusher to push.

## Putting down the wheelchair package

Follow these steps to turn the electric wheelchair into a rollator:

- a. Remove the footrests. Slide the footrests up until released from the footrest holders. Put them back into the bag of the Rollz wheelchair package, mirrored to each other (**Figure 55**).
- b. Grasp the handle of the wheelchair package and push the backrest forward until halfway up the frame. Then lower the backrest down until it folds behind the seat.

- c. Loosen the adjustment handles anticlockwise and turn the handles outwards until they point straight forward. Set the correct height for you and tighten the adjustment handles clockwise. See **Adjusting the height for correct posture**.
- d. The wheelchair package stays in place when folded and the rollator can be used to walk.

## Removing the wheelchair package

- a. Pull up the red handle without force. This way the locking bolts are tied in, and the wheelchair package unlocked (**Figure 56**).
- b. Hold the handle of the wheelchair package and pull it up without force.
- c. Then push down the red handle.
- d. The package can now be stored, for example when transporting the Rollz Motion Electric in a car. The footrests remain inside the package.

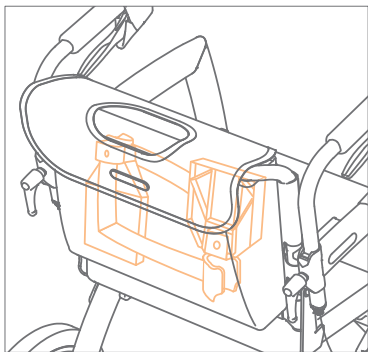
Figure 55: Storing footrests mirrored.

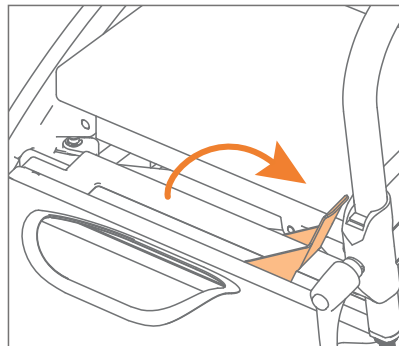
Figure 56: Unlocking the wheelchair package.

## Safety instructions

In the dark or with heavy rainfall, visibility on the road can be obstructed. Look together for the best and safest walking route. Good visibility is important.

## USING AS ELECTRIC WHEELCHAIR - with a person pushing

### Matching your pace


Whoever sits in the electric wheelchair and is being pushed for the first time, will need time to get used to it. Same goes for the person pushing the wheelchair. The person pushing and the person sitting in the electric wheelchair should regularly communicate about the pace and route. This way, both will get used to using the Rollz Motion Electric wheelchair. Walk slow at first, which will give you time to learn how to use the walker with confidence.

### Use of the electric support system - with a person pushing

While seated in the chair, the person pushing can activate the electric push support by pressing the On/Off button.

**(Figure 57):**

1. Multi-Functional display
2. Gear display
3. Battery display
4. On/Off button
5. Power assist
6. Direction
7. Gear
8. Charge port (not used)

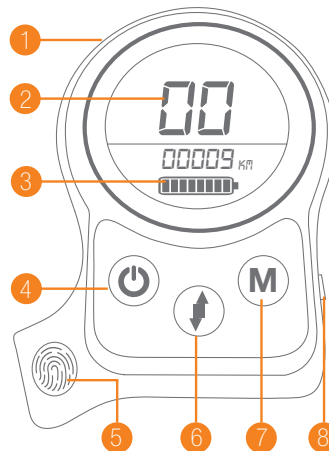**Figure 57: Push support controller buttons and components.**

The following steps can only happen when the person pushing the wheelchair has their hands on the handgrip. When the person takes their hands off the handgrip, the push support will stop.

By pushing the | **⚡** |, the person pushing the wheelchair can change the forward motion into a backwards motion.

By pushing the | **M** |, the person pushing the wheelchair can go through the different gears, which are the following:

- 01: Light power assistance, maximum speed 3.0 km/h / 1.9 mph
- 02: Medium power assistance, maximum speed 4.3 km/h / 2.7 mph
- 03: High power assistance, maximum speed 5.5 km/h / 3.4 mph

Automatic lock in mode 01 ~ 03: the automatic lock occurs 5 seconds after releasing the handgrip, then the support system will automatically lock, the display screen displays "LL", The Rollz Motion Electric locks and cannot move. Release the lock by holding the handgrip again and pressing any button or key on the push support controller, "LL" disappears and the lock is released.

## Power assist modes

**Constant power assist:** hold the handgrip, press on the On/Off button. Press with your thumb on the  button so the walker will run automatically at the set gear speed. If you want to change direction from forward to backwards, press the  button. If the LCD icon  flashes, you are in reverse mode. If you start walking backwards, a beeping sound will occur.

**Auto-power assist:** hold the handgrip, push the Rollz Motion Electric and auto-power assist automatically turns on. The more the wheelchair gets pushed, the more power the auto-power assist gives. The higher the gear, the more power. The auto-power assist works when pushing the wheelchair forwards and pulling it backwards. While going downhill, the auto-power assist can create resistance.

If the auto-power assist mode is insufficient while walking uphill, the constant power assist can be started by pressing the  button, to create a consistent walking speed.

**Anti-fall:** if the speed of the wheelchair becomes too fast while pushing the wheelchair, the wheelchair automatically brakes. In this case the display screen displays "EE". To resolve this and continue pushing the wheelchair: take your hands away from the handgrip, "EE" disappears and the anti-fall modus will ease.

## WARNING

- Do not drive into a hole or uneven terrain. The electric wheelchair might tilt with the person sitting in it.
- Do not collide hard against ledges or elevations. The person sitting in the electric wheelchair might be launched out of it.

- While walking with the electric wheelchair, look carefully ahead to determine a safe route.
- Attention: do not push the electric wheelchair off the pavement or any unevenness facing forward. This can be dangerous because the person in the electric wheelchair might topple over. Turn the electric wheelchair around and go backwards off the pavement.

### Going up a curb

It is often easier to use the dropdown of the curb of the pavement (if any). For curbs up to 15 mm:

- Place the front wheels of the wheelchair against the curb.
- Slowly push the wheelchair forwards.
- When the front wheels are on the curb, keep pushing the wheelchair forwards, so that the rear wheels also get on the curb.

For curbs higher than 15 mm and lower than the maximum curb height of 50 mm:

- Turn the wheelchair around, turn the push support off and place the wheels of the tipping aids against the curb.
- Slowly pull the wheelchair backwards so the wheels of the tipping aids go up the curb (**Figure 58**). Reverse till all the wheels have climbed the curb.
- Turn the wheelchair around and activate the push support.

### Getting off the pavement safely

To go down a curb from the pavement (**Figure 59**):

- Look carefully ahead to choose the smoothest route possible.
- Be careful when approaching curbs, height differences or uneven terrain.
- The pusher turns the wheelchair and slowly drives backwards off the curb.
- The person sitting leans against the backrest until the move is done.


#### WARNING

If the pusher suddenly moves forward with the electric wheelchair from a curb or ledge that is higher than about 15 mm / 0.6 inch, the person in the electric wheelchair can be launched forward and even fall out. This can also happen if the pusher hits a ridge at pace. Therefore, prevent yourself from driving or crashing into an obstruction.


Avoid ditches, holes or potholes. One of the front wheels can end up abruptly in a hole that causes the wheelchair to unexpectedly tip over. The disproportionate amount of force on one of the front wheels might also damage the suspension. So be careful and choose a safe route.

## Slopes

The electric wheelchair is intended for use on a hard, level surface. It is more difficult to keep a course on roads slightly banked. The slope should not exceed 3° (**Figure 60**).

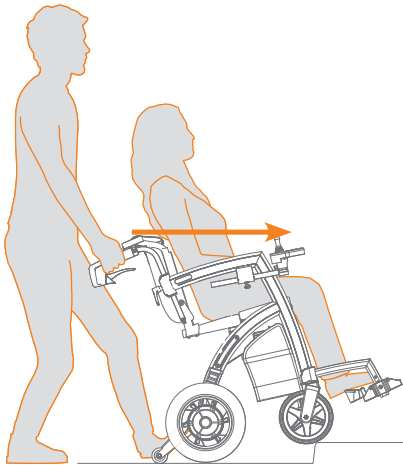**Figure 58:** Going up a curb onto a pavement.

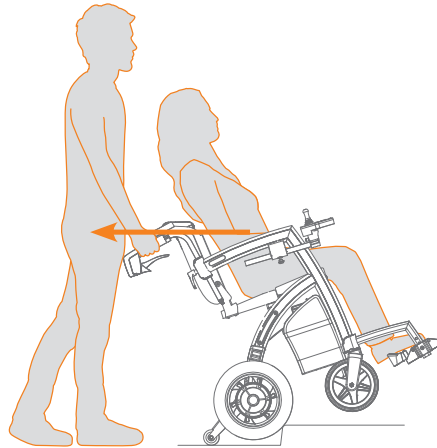
**Figure 59:** Getting off the pavement backwards.

### CAUTION


- The bank slope must not exceed 3°. If the bank angle is bigger there is a risk of tipping over with the electric wheelchair.
- If you walk up a slope, pushing can be tough. If the pusher wants to pause, apply the parking brake.
- Be very careful when walking down a slope. The pusher must brake gently to prevent it from going too fast.
- Be sure not to drive on steep slopes.

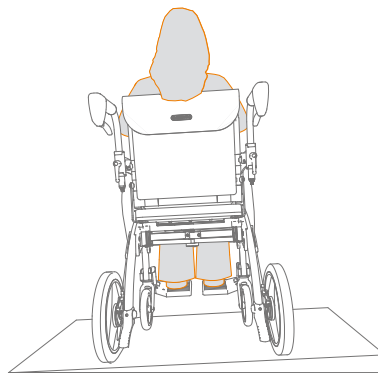**Figure 60:** The maximum bank angle is 3°.

## USE OF THE JOYSTICK - while sitting in the electric wheelchair

While seated in the chair, with the handlebars in rollator position so you can always use the handbrakes in case of emergency, so the bars above the frame, you can activate the electric support system **(Figure 61)** by pushing on the Turn On button on the front control.

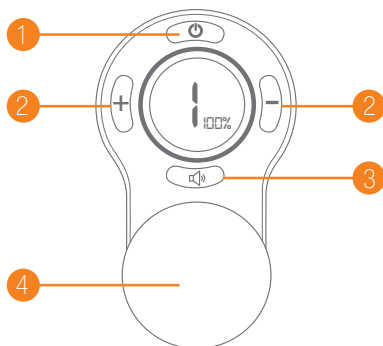**(Figure 61):**

1. On/Off button
2. Gear buttons
3. Horn
4. Joystick

**Figure 61: Joystick controller display and buttons.**

By pushing the gear buttons, | - | and/or | + | you can go through the different gears. The higher the gear, the faster the speed (1.2 – 6 km/h / 0.8 - 3.7 mph). The backwards maximum speed range is between (1.0 – 2.2 km/h / 0.6 - 1.4 mph).

By using the joystick, you can change the direction, by letting go of the joystick or moving it to the middle position, the Rollz Motion Electric will brake. You can also pull the joystick towards you, which will brake the wheelchair quicker, but this also moves the wheelchair backwards.

When using the joystick while sitting in the wheelchair for the first time, set the Rollz Motion Electric to the lowest speed. This helps you to become familiar with the control system. When you feel comfortable enough, you can choose to increase the maximum speed.

**Horn:** By pressing the |  | button a horn will sound. Use this to warn others of your approach or presence.

### **WARNING**

Be careful not to take abrupt turns at high speed. Know that the turning circle is 158 cm / 5.2 ft.

### **Reversing**

By pulling the joystick towards you, you'll drive backwards, please always drive carefully. Make sure your road is as free of obstacles as possible. Do not reverse from a slope.

### **WARNING**

- Do not drive into a hole or uneven terrain.
- Do not collide hard against ledges or elevations.
- Look carefully to determine a safe route.
- Attention: do not drive off the pavement or any unevenness higher than 15 mm / 0.6 inch.

### **Going up a curb**

To go up a curb onto a pavement (**Figure 58**):

- a. Place the front wheels of the wheelchair against the curb.
- b. Move the joystick forward, you can drive on curbs of 15 mm / 0.6 inch high.
- c. When the front wheels are on the curb, keep pushing the joystick forwards, so that the rear wheels also get on the curb.

It is often easier to use the dropdown of the curb of the pavement (if any).

### **Getting off the pavement safely**

To go down a curb from the pavement (**Figure 59**):

- a. Look carefully ahead to choose the smoothest route possible.
- b. Be careful when approaching curbs, height differences or uneven terrain.
- c. When going down a curb, slowly drive the wheelchair off the curb. The curb must be less than 15 mm / 0.6 inch.
- d. It is often easier to use the dropdown of the curb of the pavement (if any).

## Slopes

The electric wheelchair is intended for use on a hard, level surface. It is more difficult to keep a course on roads slightly banking. The slope should certainly not be too strong to prevent tilting, not more than 3° (**Figure 60**).

### CAUTION

- The bank angle must not exceed 3°. If the bank angle is bigger there is a risk of tipping over with the electric wheelchair.
- Be very careful when driving down a slope.
- Use of the Rollz Motion Electric on escalators is prohibited.

## Transfers (getting in or out of the wheelchair)

Practice getting in and out of the wheelchair with your practitioner (physiotherapist or occupational therapist) if possible.

When getting in and out of the electric wheelchair, use the armrests as a support point and not the joystick. The following points must be followed to avoid possible damage:

- Drive the Rollz Motion Electric as close as possible to the place where the transfer will take place.
- Make sure that the Rollz Motion Electric is switched off when you get in or out.
- Do not use the footrest to stand up when getting in or out.
- Apply the parking brake before getting in or out.
- Make sure that your feet do not get caught behind the footrest or frame parts.

## Seating position

To prevent tipping over while driving, you must observe a number of points that can affect the stability and balance of the Rollz Motion Electric. The following points affect the projection of the centre of gravity:

- Height and angle of the seat.
- The sitting position of the user.
- Sloping surface.
- The height and weight of the user.

- It is not permitted to make any adjustments to the Rollz Motion Electric yourself.
- Extending the arms and/or moving them to grab objects from your Rollz Motion Electric will affect the tipping point of the Rollz Motion Electric.
- Reposition yourself by leaning only on the armrests of the Rollz Motion Electric.
- Only reposition yourself while stationary.
- Do not lean on the footrest when repositioning yourself.
- Do not pick up objects from the floor that require you to reach over your knees.
- Never lean over the backrest. This can cause the Rollz Motion Electric to tip over.
- Do not lift the Rollz Motion Electric while the occupant is seated.

## SPECIFICATIONS FOR ROLLZ MOTION ELECTRIC

### Product specifications (Figure 62) - (Figure 63) Information about the material

**Frame:** welded aluminium with coating.

**Handles:** polyurethane (PU) foam.

**Seat cushion cover and wheelchair package cover:** 100% polyester.

**Inner cushion seat and wheelchair package seat:** filling of polyurethane (PU) foam.

**Joystick and controller push buttons:** synthetic rubber.

**Front wheels of the Rollz Motion Electric:** PU rubber tires.

**Rear wheels of the Rollz Motion Electric:** aluminium rims with pneumatic tires. The tire is made of synthetic rubber, the inner tube of synthetic rubber with a so-called car valve.

**Armrests:** an elastic fibre fabric of neoprene and synthetic rubber (Styrol Butadiene Rubber) filled with EVA foam rubber.


**Footrests:** aluminium frame with plastic footrests.

**Battery case:** aluminium alloy.


**Anti-tip device:** steel.

PRODUCT MEASUREMENTS	
Model	Rollz Motion Electric
Class	A
Weight rollator	19.6 kg / 43.21 lbs (excluding battery) or 22.6 kg / 49.82 lbs (including battery)
Weight wheelchair package	3.4 kg / 9.5 lbs
Weight electric wheelchair	23 kg / 50.7 lbs (excluding battery) or 26 kg / 57.32 lbs (including battery)
Rollator unfolded with joystick and push support attached H x D x W	88-99 x 71 x 74 cm / 34.65-39 x 27.95 x 29.5 inch
Electric wheelchair unfolded, with handlebars above the frame H x D x W	91-99 x 100 x 74 cm / 35.8-39 x 39.4 x 29.5 inch
Measurement rollator folded H x D x W	90 x 71 x 44 cm / 35.4 x 28 x 17.3 inch
Electric wheelchair unfolded, H x D x W	91-96 x 110 x 74 cm / 35.8-37.8 x 43.3 x 29.5 inch
Rear wheels diameter	12.5 inch (air tire)
Rear wheels width	2 ¼ inch
Front wheels diameter	8 inch (Air tire)
Front wheels width	1 ¼ inch
Reversing width	116 cm / 45.7 inch
Range	15 km / 9.3 mile
Max speed	6 km/h / 3.73 mph
Stopping distance	<100cm / <39,4 inch
Sound level	<65 dB

The distance range will be reduced if the wheelchair is used frequently on slopes, rough terrain or to climb kerbs. The stopping distance on slopes can be significantly greater than on level ground. The maximum width of 740 mm / 29.1 inch is larger than the maximum width recommended in a.1.1 of EN 12184:2014 of 700 mm / 27.6 inch. Access to escape routes can be limited.

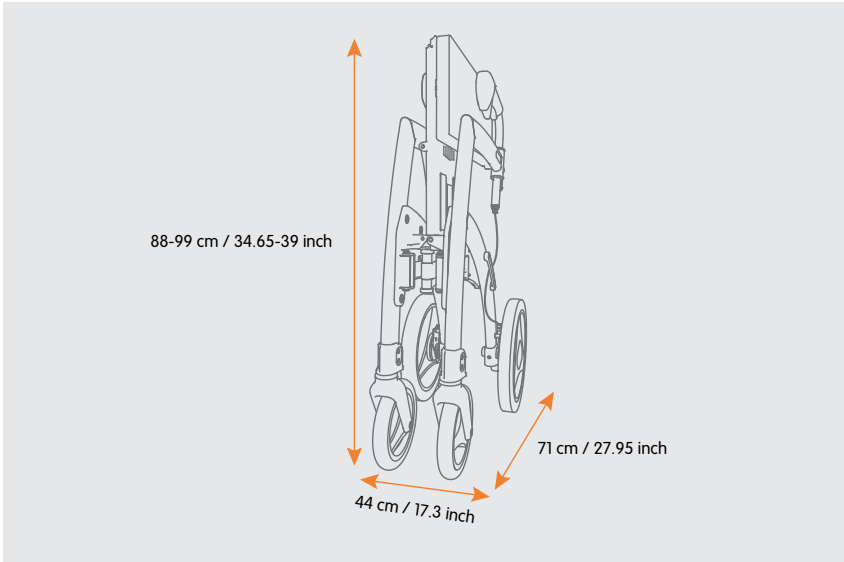**Figure 62:** This is how compact the folded Rollz Motion Electric is.

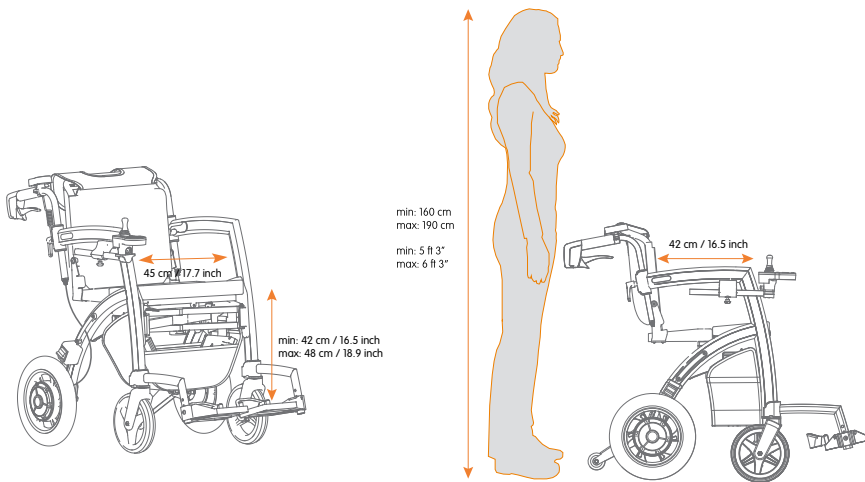**Figure 63:** Ergonomic dimensions of the Rollz Motion Electric electric wheelchair.

ERGONOMIC DIMENSIONS	
Rollz Motion Electric	
Maximum weight of user	100 kg / 220 lbs
Minimum and maximum height of the user and pusher	Ca 1.60 m till 1.90 m / 5.25 ft till 6.25 ft
Seat of rollator H x D x W	55 x 20 x 45 cm / 21.7 x 7.9 x 17.7 inch
Seat of electric wheelchair H x D x W	55 x 42 x 45 cm / 21.7 x 16.5 x 17.7 inch
Height handles rollator	88-99 cm / 34.25-38.6 inch
Height handles electric wheelchair	84-96 cm / 33.1-36.61 inch
Distance between handles	44.5 cm / 17.52 inch

TECHNICAL DETAILS	
Rollz Motion Electric	
Motor	DC24V250W2
Motor power	2*250 W
Battery type	Li-ion
Battery capacity	12,5 Ah
Battery nominal voltage	24 V
Charger and maximum output current	AC100-240 V 50-60 Hz and 2 A output
Controller maximum output current	40 A, running current 2-3 A

## ERROR CODES AND SOLUTIONS

An error code can appear when an error occurs. This is indicated when 'EE' is displayed on the LCD screen. Below 'EE', the specific error code will be shown. Find the error code below to read the recommended solution.

ERROR CODE LISTING		
Error code	Failure cause	Solution
E0002	Left motor malfunction	Check whether the left motor cable and connector is damaged. If damaged, contact the dealer to replace broken parts. If the outlet or cable is not damaged, check whether the left motor cable is securely inserted. If this is correctly inserted, contact the dealer to look into this error code further.
E0004	Right motor malfunction	Check whether the right motor cable and connector is damaged. If damaged, contact the dealer to replace broken parts. If the outlet or cable is not damaged, check whether the right motor cable is securely inserted. If this is correctly inserted, contact the dealer to look into this error code further.
E0008	Left motor current detection error	Contact manufacturer to replace the broken parts.
E0010	Right motor current detection error	Contact manufacturer to replace the broken parts.
E0020	Battery power detection is incorrect	Contact manufacturer to replace the broken parts.
E0040	15V voltage malfunction	Contact manufacturer to replace the broken parts.
E0800	Front control communication failure	Check if the cables and connections are correctly connected, see <b>Cables and connections (Weekly)</b> . If connected correctly, contact manufacturer to replace the broken parts.
E-021	Rear control communication failure	Contact manufacturer to replace the broken parts.
E-023	Attitude sensor abnormal	Contact manufacturer to replace the push support controller.
E-024	Vehicle attitude calibration	Make sure the handlebars are in push support position. Push and hold the On/Off button until you hear a voice prompt, release the button and keep the Rollz Motion Electric stationary until the 00001 disappears from the screen.
E-025	Grip detection sensor error	Contact manufacturer to replace the broken parts.
E-026	Grip detection sensor error	Contact manufacturer to replace the broken parts.
E-027	The touch panel is abnormal	Contact manufacturer to replace the broken parts.

## ACCESSORIES

The functionality of the Rollz Motion Electric can be expanded by using accessories such as a travel cover, umbrella, belt, cup holder and flexible back support. Use only original or Rollz International BV approved accessories. New accessories are regularly released. Visit [www.rollz.com](http://www.rollz.com) or ask your supplier for new accessories.

## MAINTENANCE

Regular maintenance extends the life and increases the safety of your Rollz Motion Electric. Only let authorised Rollz dealers replace or fix parts that are worn, damaged or broken.

### Cleaning/disinfecting

Clean your Rollz Motion Electric frequently to keep it in optimal condition.

- a.** Clean the frame and handgrips using lukewarm water and a cloth. Thoroughly dry your Rollz Motion Electric after cleaning. For more resistant stains you can use a mild detergent.
- b.** Avoid excessive water contact with moving parts, such as axles and hinge points.
- c.** The handles of the rollator, the handle of the wheelchair package and the armrests may not be disinfected with an alcohol-based disinfectant. This will damage the material. Clean these parts with warm soapy water. Make sure to dry everything to prevent corrosion.
- d.** The wheels can be cleaned using lukewarm water and a brush with plastic bristles. Do not use a wire brush because it can damage your Rollz Motion Electric.
- e.** Before passing the Rollz Motion Electric on to a new user, it must be disinfected.
- f.** Do not rinse the electrical parts with water. NOTE: When the battery has come in contact with water, turn the product completely off before removing the battery, and don't use the Rollz Motion Electric before drying the battery. After the Rollz Motion Electric is completely dry, you can replace the battery.


### NOTE

Do not use an alcohol-based disinfectant.


## Disconnecting the battery

Turn off the system, apply the parking brake, do not have a person seated in the wheelchair and position the Rollz Motion Electric on a flat surface before removing the battery.

Find the battery cable and disconnect the battery cable by turning the metal ring on the connector anticlockwise. You can find the battery cable in the basket under the seat. After disconnecting the cable you can unwrap the Velcro strips to take out the battery.

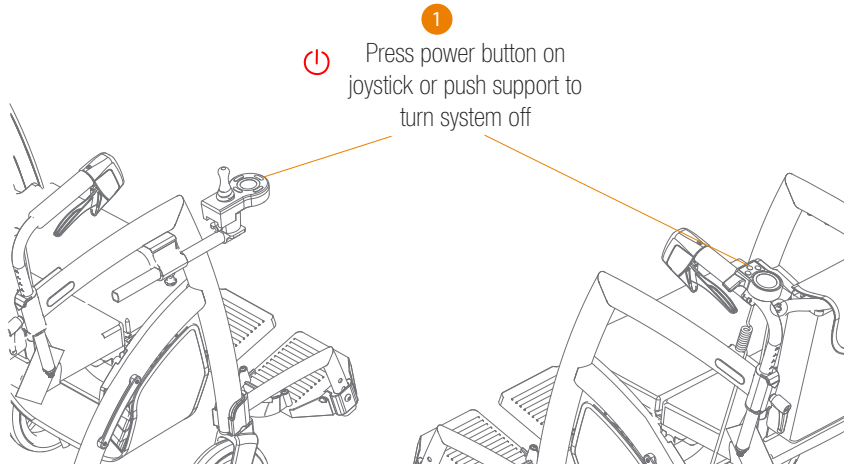
**Figure 64:** Turn the system off before removing the battery.

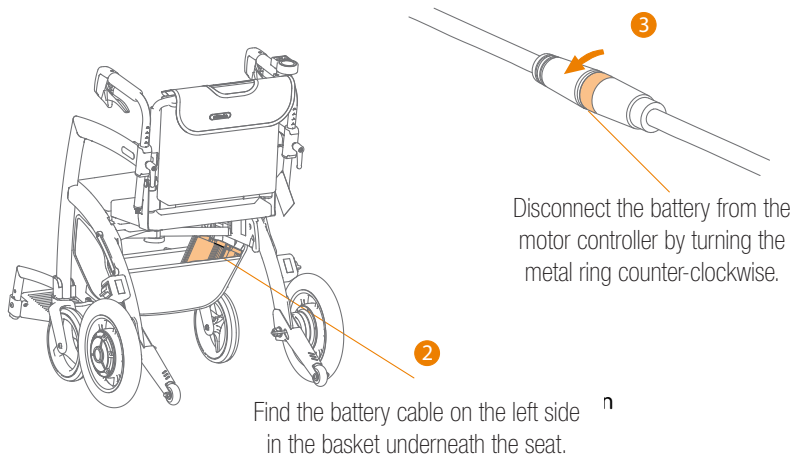**Figure 65:** Locate and disconnect the battery.

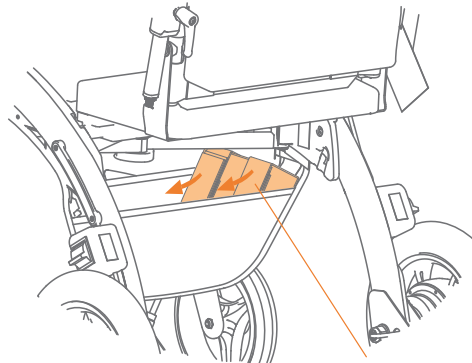
Unwrap the battery.

4

**Figure 66: Unwrap the battery.**

### Reconnecting the battery

Place the battery in the bag with the cable pointing downwards. Wrap the Velcro strips around and tighten these firmly. Find the battery cable and connect to the battery by turning the metal ring on the connector clockwise. You can find the battery cable in the basket under the seat. After checking the cable connection you can power the system back on and test the functionalities of the system.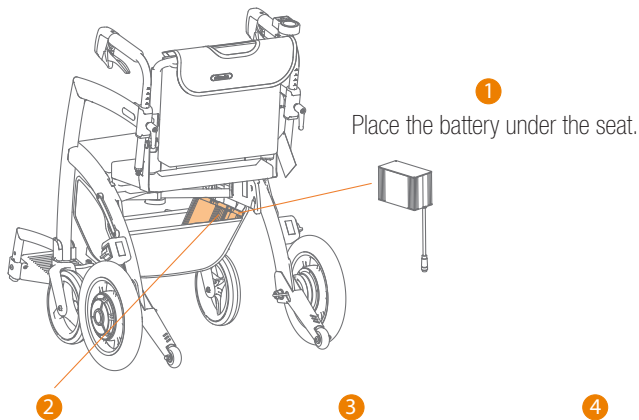

Place the battery under the seat.

2  
Reposition the Velcro straps and re-connect it with the motor controller.

3  
Press the power button and turn the system back on.

4  
Check all functions.

**Figure 67: Reconnect the battery.**

## Storing the Rollz Motion Electric

Always store your Rollz Motion Electric in a clean, dry and dust-free environment.

Do not store your Rollz Motion Electric in an environment where it may come into direct contact with water; temperatures lower than -5 °C / 23 °F and higher than 45 °C / 113 °F; with very high humidity.

## Battery maintenance (Daily)

Check the remaining battery capacity. If the battery reaches the end of its service life and should be replaced, please contact the supplier or an authorised Rollz dealer.


Battery life and performance depend on temperature, battery status and user weight. As the battery ages, a loss of capacity can occur (if there's a significant loss of range or charging the product takes significantly longer). In this case we recommend replacing the battery by contacting the dealer.

Before use, check that the battery is fully charged. The normal charging time of the battery is around 5 hours. When the battery is completely empty, the charging time can be up to 6 hours.

If you do not use the battery for a longer period of time, make sure that it is charged at least every 2 months. Failure to charge the battery for an extended period of time may cause internal damage.

## Cables and connections (Weekly)

Check the electrical parts and connecting wires for proper connections and for any damage. Even the smallest damage can lead to dangerous situations (e.g. short circuit) and will need to be fixed immediately. If damaged, please contact the supplier or have an authorised Rollz dealer repair it. Do not repair it by yourself. Check if the connections have been fixed according to **Figure 68**. Do not remove or disconnect components while having the electric support system turned on.

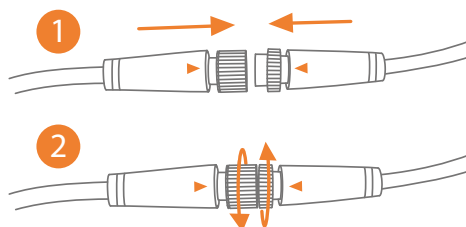**Figure 68:** Make sure that the connectors are aligned properly and tightened if applicable.

### Motors (Every month)

Inspect the motors in the rear wheels for visible cracks or deformations, check if the wheels can rotate freely and check for increased noise during use. Verify that the nuts and other fasteners that attach the motors to the frame are tightened properly and have no slack. Do not use the Rollz Motion Electric if any of these issues have been noticed. Contact the supplier or an authorised Rollz distributor.

### Push support controller and joystick controller (Every 3 months)

Clean the controller and joystick with a cloth dampened with a neutral diluted cleaner carefully. Never use abrasive materials or alcoholic detergents for cleaning. Protect the controllers from damage during transportation of the wheelchair. Always prevent liquids ingress in openings of the controllers.

### Washing the seat cover

The cover of the seat cushion is washable. Always remove the inner cushion before washing.

- a. Pull the Velcro on the cover of the cushion off from the seat **(Figure 69)**.
- b. Unzip the cover and remove the inner cushion.
- c. The cover is now ready for washing in the washing machine at a maximum temperature of 30 °C / 86 °F.
- d. Simply dry the cover in the air, not in the dryer.
- e. Put the inner cushion back in the cover and zip up the seat.
- f. Place the cushion back on the seat.

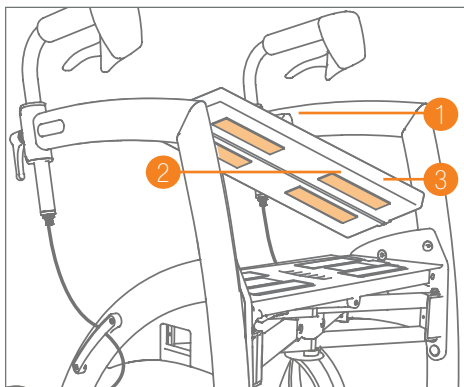
Figure 69: Removing the seat cushion from the rollator.

#### NOTE

Never bleach or dry-clean the cover and do not iron it.

#### What is where?

1. Seat cushion
2. Velcro
3. Zipper


## Brake performance (Daily)

Check the braking performance. When the brakes are in use, the wheels of the Rollz Motion Electric should come to a stop. If the wheels do not stop when using the brake: immediately shut down the system, check for damage and try to adjust the brakes according to the explanation below. Contact the supplier if the brakes are damaged or when the braking performance does not allow the Rollz Motion Electric to stop after adjusting. Do not repair the brakes by yourself.


## Adjusting the brakes

By using the Rollz Motion Electric, the brakes can wear out. As a result, it may be necessary to readjust the brakes. When the brakes are correctly adjusted, you prevent problems and can make optimal use of them. Check the brakes regularly, preferably before each use. When the Rollz Motion Electric has the parking brakes applied, the wheels should not turn. When the hand brakes are released, the Rollz Motion Electric should not be slowed down.

If the brakes do not work properly, they can be adjusted in two ways: at the top and bottom of the brake cable (**Figure 70**). The adjustment at the top of the brake cable (just below the height adjustment handle) is for fine-tuning, so the performance of the brake can be set a little tighter. The adjustment at the bottom of the brake cable (near the rear wheels) is for a more radical adjustment. Make sure that both the left and right brakes are properly adjusted.

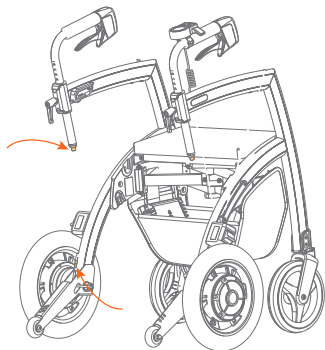**Figure 70:** Brake adjustment via the adjustment screws at the top and bottom of the brake cable.

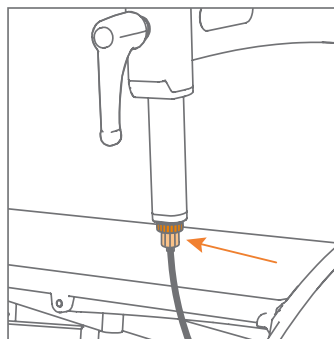**Figure 71:** Brake adjustment via the adjustment screw, just below the locking ring.

## Brake adjustments at the top of the brake cable

You can tighten the brakes using the brake adjustment at the top.

- a. The Rollz Motion Electric must not be on the parking brakes while adjusting the brakes.

- b. First loosen the locking ring (pull the cable down to make it easier). This gives you the space to tighten or loosen the adjustment screw under it **(Figure 71)**.
- c. Tighten the locking ring again and check if the brake provides the desired braking force.
- d. Do the same on the other side and check if both brakes offer the same resistance.

### Brake adjustments at the bottom of the brake cable


If you need to tighten the brakes more than what the top adjustment will allow, you can further adjust using the bottom brake adjustment.

- a. The Rollz Motion Electric should not be on the parking brakes while adjusting the brakes.
- b. Stay at the rear of the Rollz Motion Electric when adjusting the brakes. Loosen the locking ring at the bottom of the brake cable (pull the cable up to make it easier). Turn the adjustment screw above it to shorten or extend the brake cable **(Figure 72)**. Turn clockwise for more braking power, and anticlockwise for less braking power.
- c. When the brake has the desired force, tighten the locking ring.
- d. Do the same on the other side.
- e. Check whether the braking force on the left and right side is evenly distributed.

#### CAUTION

If the brakes do not function properly after adjusting them, contact your supplier immediately and do not use the Rollz Motion Electric.

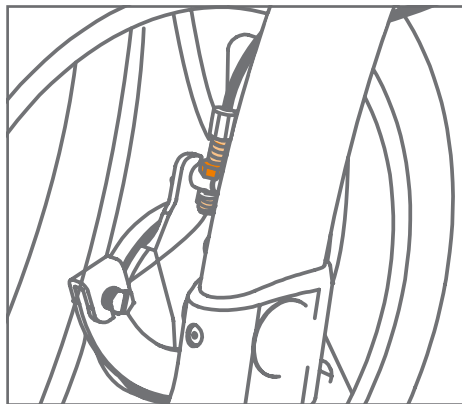
**Figure 72:** Brake adjustment via the circlip (dark orange) and the adjusting screw.

## Inflating the pneumatic tires (approximately every month)

The Rollz Motion Electric has pneumatic rear wheels. These pneumatic tires need to be provided with extra air regularly for optimal usage (**Figure 73**). The inner tubes are equipped with Schrader valves, also known as car valves. Always bring the included pump with you on the road to use it if necessary. It fits in the bag of the wheelchair package.

About two pumps per band are usually enough to get the recommended pressure. The rear tires must have a maximum pressure of 2.5 bar ( $\approx$  2.5 atmospheres or 36.3 PSI / or 250 kPa). The front tires must have a maximum pressure of 4.5 bar ( $\approx$  4.5 atmospheres or 65.3 PSI or 450 kPa)

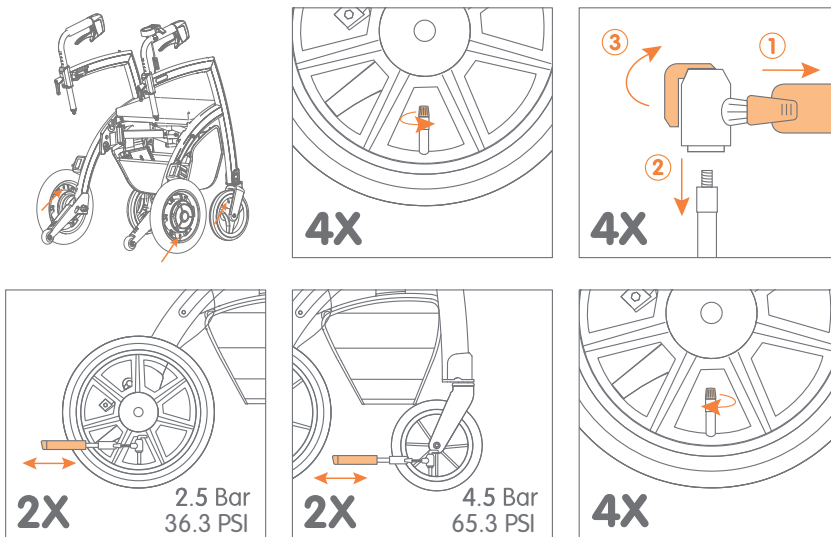**Figure 73: Inflating the pneumatic tires.**

### CAUTION

Do not inflate the tires over the advised pressure because they could explode under high pressure.

### NOTE

Frequently check if the tires are inflated up to the recommended pressure. If the tires are too soft, the Rollz Motion Electric will be harder to push.

## Flat tires

Immediately turn the system off and do not use the Rollz Motion Electric if a tire is flat, punctured or broken. Contact the supplier or authorised Rollz dealers to replace the tire.

## Replacement of parts

Wearable parts, like foam handles, tires, wheels, brake cables and the seat cushion, are subject to wear and tear. Please contact your supplier for the replacement of parts if necessary.

### NOTE

Only use original or Rollz International BV approved parts.

## WARRANTY

Is something wrong with your Rollz Motion Electric within two years of purchase? Then contact your supplier immediately. They will look for a solution as quickly as possible in consultation with Rollz International. We apply the warranty provisions below.

Repairs are also possible after the warranty period has expired, against reimbursement of costs incurred. Please contact your dealer directly for this.

## Warranty provisions

Rollz® International BV, established in Delft in the Netherlands (hereinafter "Rollz"), in its capacity as manufacturer, guarantees the Rollz Motion Electric (hereafter: "product") are produced under the following provisions:

1. The general warranty period for the first owner (hereinafter: "customer") is 24 months from the date of purchase.
2. Rollz gives a five-year guarantee from the date of purchase on material and construction errors of the frame during normal use and maintenance.
3. The battery is under warranty for a period of 18 months.
4. The electric support system, except the battery, is under warranty for a period of 24 months.
5. The warranty gives the right to repair, explicitly not to exchange or to take back a product.
6. In case of repair, the customer takes care of the transport of the product to and from the store where it was purchased (the "supplier"). The supplier will arrange the repair in consultation with Rollz.

7. The warranty can only be invoked if the customer provides the supplier with a copy of the purchase receipt with purchase date and a clear description of the complaints, together with the defective product.
8. If necessary, Rollz takes care of the transport of the product from the supplier to Rollz and vice versa for its own costs and risk, but only if the repair is covered by the warranty.
9. The warranty period is never extended, even in the case of repairs.
10. The warranty is not transferable and can therefore only be invoked by the first owner.
11. The warranty does not affect the rights that the customer has under the law.

The warranty does not apply in the following situations:

- a. if the customer has made or ordered changes or repairs without prior written permission from Rollz or the supplier;
- b. when defects have been caused by neglect or an accident and / or use or maintenance, other than what is stated in the manual;
- c. if the serial number is damaged or removed;
- d. if the damage to the product is the result of normal wear and tear.

### **Register your Rollz Motion Electric**

Please register your product with Rollz International by filling out the online form on [www.rollz.com](http://www.rollz.com). Make sure to enter the serial number of the Rollz Motion Electric. This number can be found on the sticker on the seat (under the cushion). When registering, you can also sign up for an e-mail newsletter to stay informed about Rollz product news.

## CONTACT

Rollz International is always keen to further improve its products, so therefore we are interested in your opinion. This could be about the use of the product, the manuals or any other aspects.

Please send your tips or experiences to: [info@rollz.com](mailto:info@rollz.com).

Or contact us for any other questions or comments you may have:

### **Rollz® International BV**

Rotterdamseweg 402M

2629HH Delft

The Netherlands

+31 (0)20 362 20 10

[info@rollz.com](mailto:info@rollz.com)

[www.rollz.com](http://www.rollz.com)

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De rollator en elektrische rolstoel ineen

rollz motion electric

NL

Nederlands

## Rollz® International BV – Rollz Motion Electric

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