

LET'S DREAM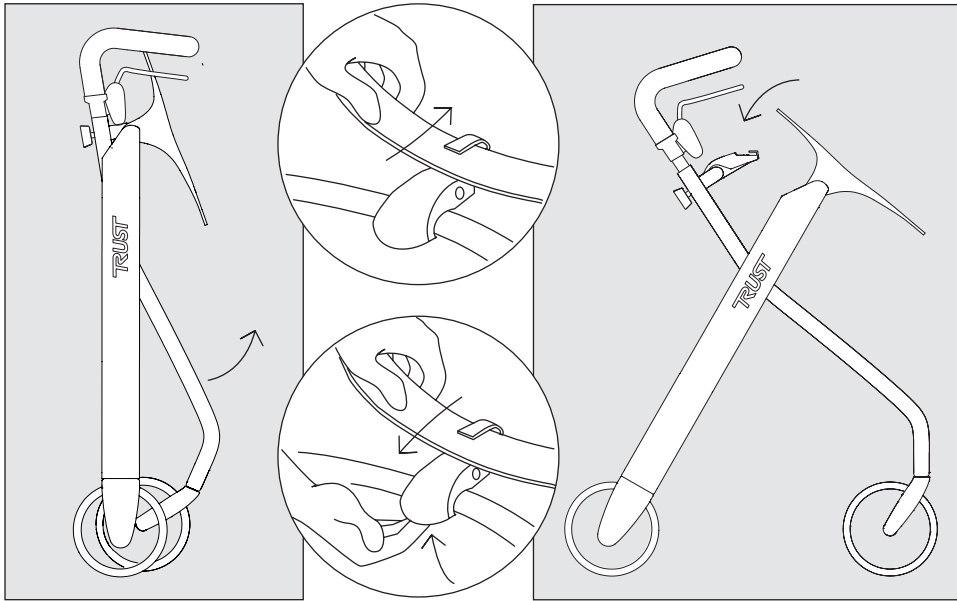


1.





2.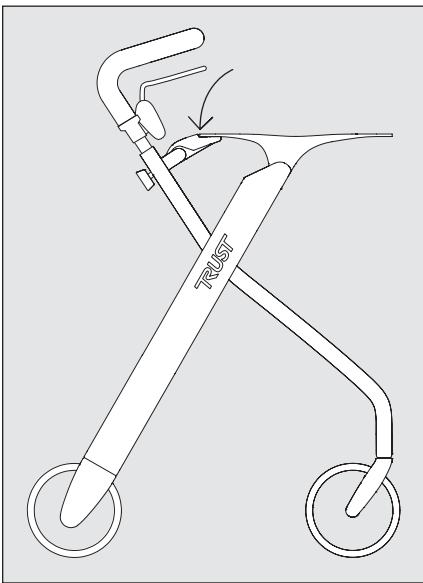

3.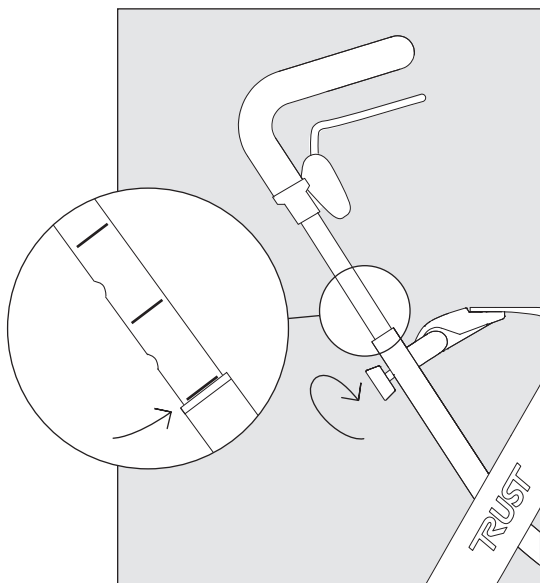

4.

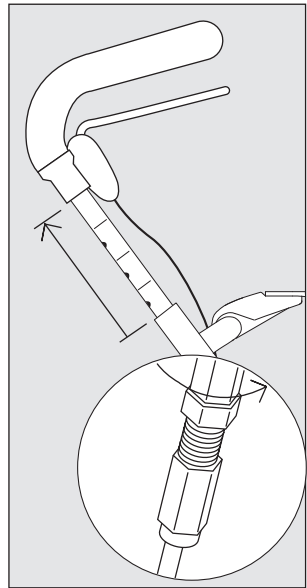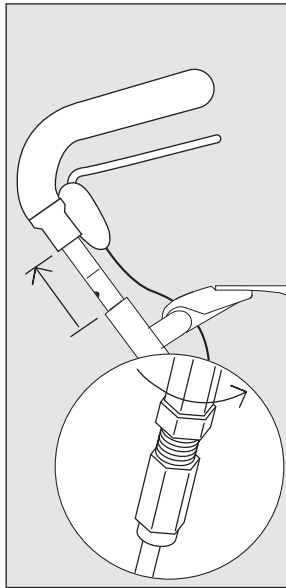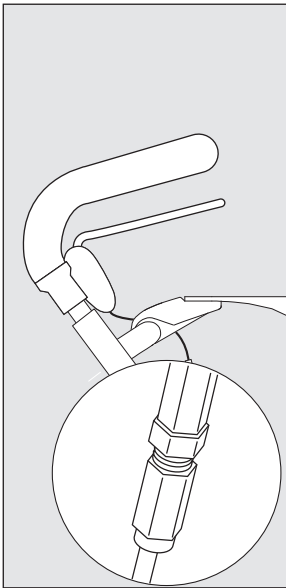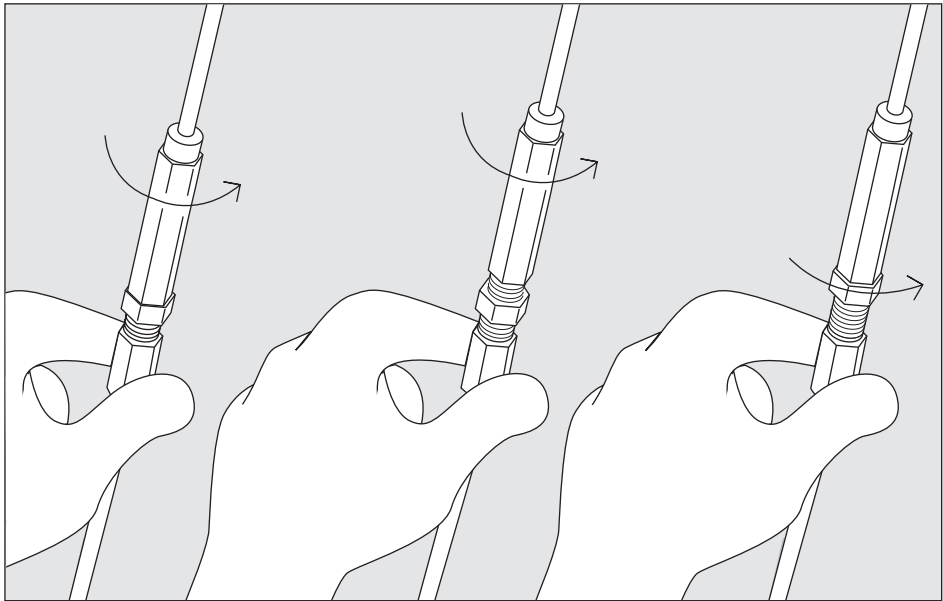SVENSKA

Let's Dream Inomhusrollator.

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ENGLISH

Let's Dream Indoor Walker.

Congratulations to you for purchasing your new indoor rollator that will make your life a little easier, more pleasant and more safe.

To make it easier for you to get familiar with your rollator it is important that you study this manual.

The rollator helps you to walk and move in-doors. You can use it as a small sideboard when you are sitting in a couch or an armchair, or place it in front of you using it as a small table when you eat or drink your coffee. Do not forget to activate the parking brake – position 3 on the picture to the right. See picture on page 15 of this manual.

When crossing thresholds, be careful so the items you have placed on the tray do not fall. Avoid hot liquids in open vessels.

The tray is designed to carry only light and small objects like a plate, a coffee-cup, a watering-can etc. Do not move heavy objects like large boxes, pieces of furniture etc.

The rollator is not intended to sit on. It is designed only to carry the tray and the special fabric basket that can be obtained as accessory. The basket is intended for light objects like fruit, yarn, a thermos and such.

Maintenance: Clean the rollator and the tray with water and a little detergent.


Warranty:

We provide a one year warranty. It does not cover wear parts like wires and wheels.

How to unfold.


(See pictures on page 2!)

1. Rollator standing. Unfold the front wheel frame by moving the two front legs forward.
2. Let the tray support ring slide into the hook from backwards.
3. Make sure that the hooklock snaps into place and locks.
4. Loosen the two adjustment screws. Adjust the handle and tighten screws.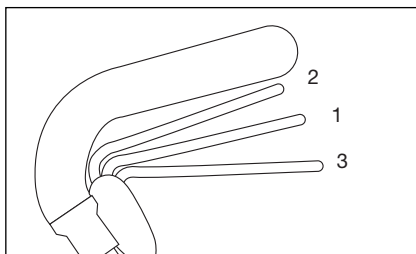

Brake functions.

1. In position 1 the rollator is rolling freely.
2. To brake the rollator pull the handle to position 2.
3. For parking brake push handle down to position 3.

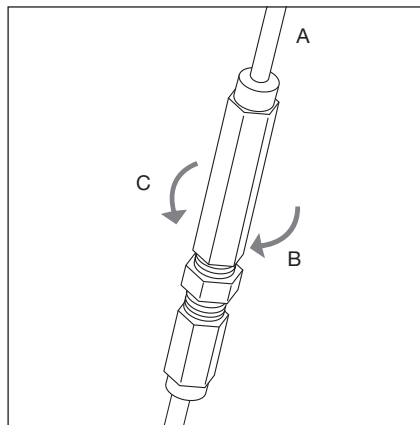Brake adjustment.

1. The brakes are adjusted with the adjustment screw on the wire (A).
2. Adjustment of the wire tension is done by turning the adjustment screw (B-C).

Technical data:

Tested according to ISO 11199-2 for a maximum user weight of 100 kg.

Max. width 55 cm.

Weight 6,5 kg.

Handle height adjustable from 83 to 95 cm.

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