---

GEBRAUCHSANWEISUNG  
USER'S MANUAL

---

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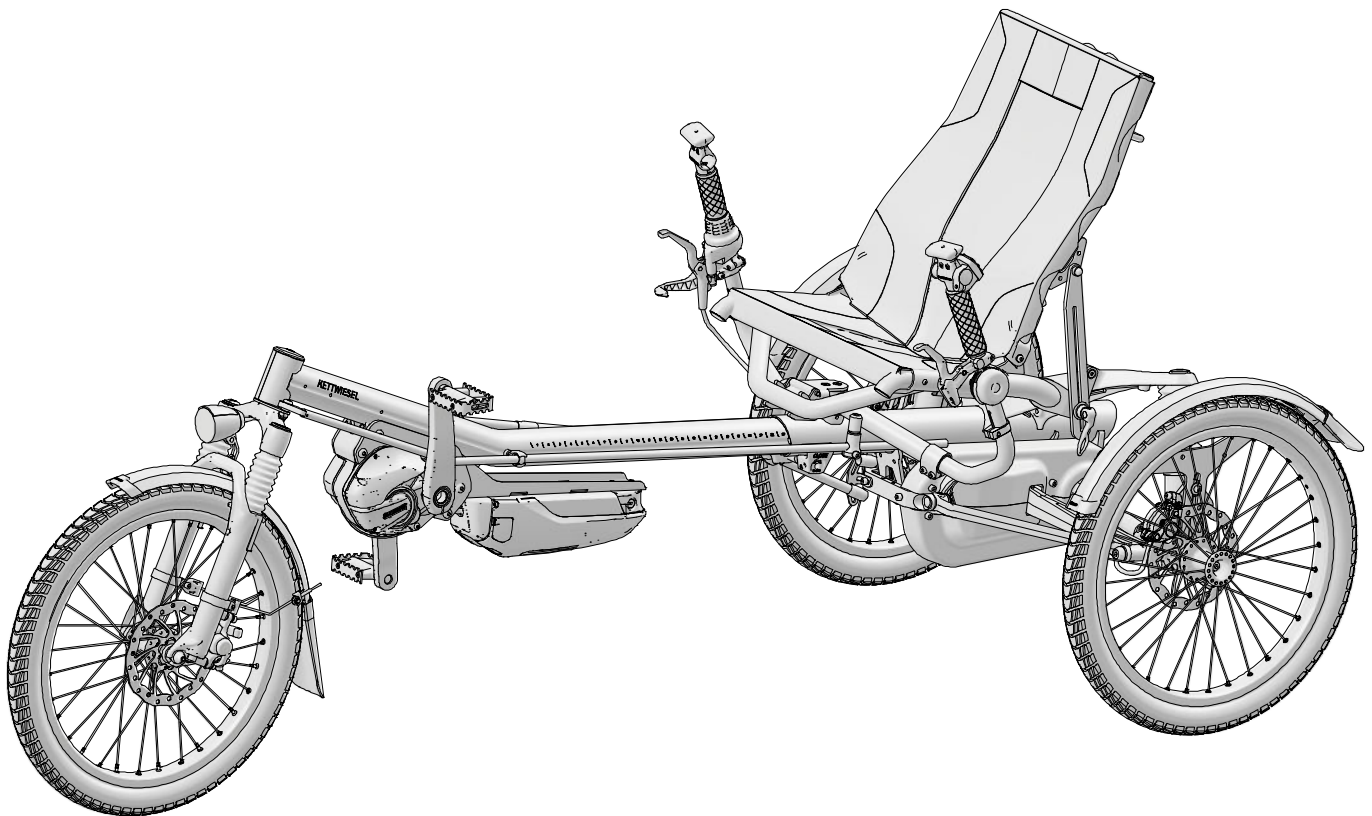
# Kettwiesel ONE / Kettwiesel ONE UP

Kettwiesel\_2024\_DE-EN

02 - 25

**REF** Artikel /  
Article

27649 ; 28269 ; 28389

Komponenten

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## What is the Kettwiesel ONE (UP)?

The Kettwiesel is a recumbent trike and an approved medical device as per the Medical Devices Regulation (EU) 2017/745 (MDR). The trike is driven by leg power via the pedals.

An internal gear hub transfers torque to both rear wheels. The padded seat, in combination with the full suspension, offers you maximum riding comfort. The trike is equipped with two independent brake systems.

## Intended use

The Kettwiesel is suitable for children, teenagers and adults between 1,1 and 2 meters tall.

The Kettwiesel is intended for use on paved roads and bicycle paths, on roads or trails surfaced with sand, gravel, or similar materials (e.g. forest paths, dirt roads), and on surfaced and unsurfaced forest trails, where tree roots, bumps, stones, and steps are more common, although the wheels must remain in contact with the ground at all times unless riding over obstacles like tree roots, when they may temporarily lose contact with the ground.

## Indications for use

The Kettwiesel is suitable for riders with neuromuscular disorders, such as cerebral palsy or muscular dystrophy, who owing to special physical conditions are unable to ride standard bicycles or vehicles, even with training wheels. Other indications include coordination disorders, balance disorders, and restricted movement associated with spasticity, paraplegia, dysmelia, amputations, etc.

## Contraindications

The Kettwiesel is not suitable for people with visual impairments. In tandem operation, the Kettwiesel can under certain conditions be used by people with visual impairments.

## Adaptability to individual needs

The Kettwiesel can be equipped to accommodate riders with various physical impairments. You can find optional accessories at [www.hase-bikes.com](http://www.hase-bikes.com) or ask your Hase Bikes dealer for advice.

## Service life

Depending on the frequency of use, a service life of up to 8 years can be expected if the product is used in accordance with the user manual.

## Reuse

Under normal conditions of use, the Kettwiesel can be reused by a new owner after cleaning, disinfection and by inspection by an authorized dealer, observing the maintenance and inspection table.

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## Protection against environmental influences

Owing to the materials used for its components, e.g. coated and anodized aluminum frame parts, plastics, and stainless steel, the Kettwiesel is corrosion-resistant.

## Cleaning and disinfection

The handlebar grips and seat can be cleaned and disinfected using household cleaning agents and disinfectants. You can use dishwashing liquid, for example.

Clean the trike regularly. Dirt or salt from winter use could cause damage. For cleaning, it is best to use water with a biodegradable household cleaning product and a sponge or towel. Never use pressure washers. Never use waxes, cleaning products or protective coatings on brake and gear components, grips, chains, seat covers, and tires. Lubricate the chain after cleaning or long rides in the rain.

## Applicable standards and regulations

The Kettwiesel has been designed and manufactured in compliance with the following regulations and standards: Medical Device Regulation (EU) 2017/745 (MDR).

## Disposal instructions

At the end of its useful life, the product should be disposed of properly, in accordance with the local waste disposal regulations (i.e. not as household waste).

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## Read the instructions manual

Read this user's manual and the manuals for the associated components carefully, and familiarize yourself with the vehicle in practical use. For persons with visual impairments, these manuals are available in enlarged form on our website.

## Before your first ride...

Your authorized retailer has assembled your new trike and adjusted it to your size requirements. Please refer to the section "Adjust to body size" on page 14 if you need to make adjustments for another rider. Before setting out on your first ride, please take some time to familiarize yourself with the trike.

The brake pads on a new trike must first be broken in to ensure optimum braking performance. Due to the Kettwiesel's unique construction, there are significant differences in handling in comparison with a standard bicycle. Before riding the trike on public roads, it is important to familiarize yourself with the handling in a quiet, open area away from traffic.

The maintenance measures required before each ride are to be performed by the user (individuals with no physical, mental, or visual impairments).

## Observe traffic regulations

Traffic regulations vary from country to country. For example, the trike may have to be equipped with a lighting system, headlights and/or a bell to comply with the respective regulations. We offer accessories in accordance with the German Road Traffic Act (StVZO).

Consult your authorized retailer for information on the accessories required. All equipment should be installed by a qualified bike mechanic.

If you plan on installing or making changes to the lighting system yourself, please study the applicable regulations before you begin.

## Being visible for others

The Kettwiesel is lower than an upright bicycle. This may make it more difficult for other road users to see you. Therefore, we recommend riding with a flag at all times.

## Riding at night

Always use lights when riding at night or in conditions of low visibility, such as dusk and dawn.

## Avoiding muscle strain

In a recumbent position, the rider uses different muscle groups than with a standard bicycle. Riders with little or no experience on recumbent bikes/trikes should begin with short, easy trips in order to avoid excessive muscle strain.

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## **Suitable clothing and helmet**

Do not wear loose clothing that could get caught in the wheels or gears. Beware of scarves and clothing strings/cords that hang down. Always wear a helmet in order to reduce the risk of head injuries in the case of an accident.

## **Risk of injury from rotating parts**

If a rider seated in a trike that is being pulled as a trailer allows their hands or fingers to hang down beside or below the seat, there is a risk that they could get caught in the disc brakes and/or wheels. To prevent the associated injuries, it is essential to ensure that these riders keep their hands/fingers on the handlebar grips or above the seat at all times!

## **Pedals**

In a recumbent position, it is especially easy for the rider's feet to slip from the pedals. Always wear shoes with good tread. Maintain sufficient pressure on the pedals. For children, pedals with toe clips and straps (slip protection) are available as optional accessories.

## **Technical condition**

Before each ride, perform a visual inspection to look for damage. Take the Kettwiesel to your bike shop for regular maintenance. The recommended maintenance intervals can be found in the section "Maintenance and Inspections."

## **Brakes**

The trike is supplied as standard with two rear brakes or with a coupled rear brake and front brake. Both brake handles act separately on the right and left rear brakes or, if a front brake is also fitted, on the front and rear brakes.

The two braked rear wheels offer significantly greater deceleration than the front brake.

When riding on steep terrain, the front and rear brakes should be operated simultaneously.

The built-in disc brakes are very powerful and easy to control, but the wheels can still lock up.

Before riding on the road for the first time, familiarize yourself with the braking force by testing the brakes on an untravelled, level surface. Determine in advance which brake lever actuates which brake.

## **Checking the disc brakes**

Observe the brake manufacturer's operating instructions. Have the pad thickness checked regularly by your specialist workshop. If the braking effect is insufficient, contact your specialist workshop immediately. New brake pads must first be braked in to ensure optimum braking performance.

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## Maximum load

Please note that the total weight of the rider and cargo must not exceed 140 kg (309 lb). Only transport cargo at the designated sites. Ensure that the load is evenly distributed.

## Jumps and competition use

In general, jumps of up to 0.2 m (8") are permitted, but at the rider's own risk. Your riding safety on roads, on paths and off-road will depend on your speed. The higher your speed, the greater your risk! The use of the Kettwiesel for competitive purposes is strictly forbidden.

## Serial number and bike coding

The frame number is located at the front under the front boom. This is also the only site where a bicycle code may be attached. Bike coding has been shown to help identify the owner in the event of theft or loss.

## Risk of flammability

The upholstery material of the seat cover is not resistant to ignition sources as specified in EN 1021-2. Therefore, be sure to keep the seat away from open flames, heat sources, etc.

## Children

Children may only use the Kettwiesel under parental supervision.

## Solar radiation/Frost

Please note that plastic and metal parts exposed to direct sunlight can heat up and cause skin injuries if touched.

Also avoid exposing the product to frost for long periods of time. Metal parts that have cooled down considerably can cause injuries on contact with the skin.

## Accessories

Only use original parts and accessories from Hase Bikes.

## Further information

If you have any problems or questions about your therapeutic bike or trike, please contact your authorized dealer or HASE BIKES for assistance. Any serious incident\* that occurs in connection with the product must be reported to the manufacturer and the responsible competent authorities of the member state in which the user and/or patient resides.

\*A "serious incident" is any incident that had, could have had, or could have one of the following consequences, either directly or indirectly:

- a) the death of a patient, user, or other person,
- b) the serious temporary or permanent deterioration of the health condition of a patient, user, or other person,
- c) a serious risk to public health.

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### General information

#### Torque Specifications

Use a torque wrench for all bolts. If not specified in the instructions, the table gives the guide values for all screws of 8.8 quality.

| Screw                    | Torque             | Screw | Torque               |
|--------------------------|--------------------|-------|----------------------|
| M3                       | 1.2 Nm (0.9 ft-lb) | M6    | 9.0 Nm (6.6 ft-lb)   |
| M4                       | 2.9 Nm (2.1 ft-lb) | M8    | 24.0 Nm (17.7 ft-lb) |
| M5                       | 5.5 Nm (4 ft-lb)   | M10   | 46.0 Nm (34 ft-lb)   |
| Clamp main frame         | 14 Nm (10.3 ft-lb) |       |                      |
| suspension arm (M8x95)   | 18 Nm (13.3 ft-lb) |       |                      |
| Suspension strut (M6x25) | 5 Nm               |       |                      |

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### Quick-release levers

**Opening:** Swing the lever (01) 180° up and over to the open position (the word “OPEN” will be visible). In order to open the quick-release mechanism even further, brace the tension-adjusting nut while turning the quick-release counterclockwise.

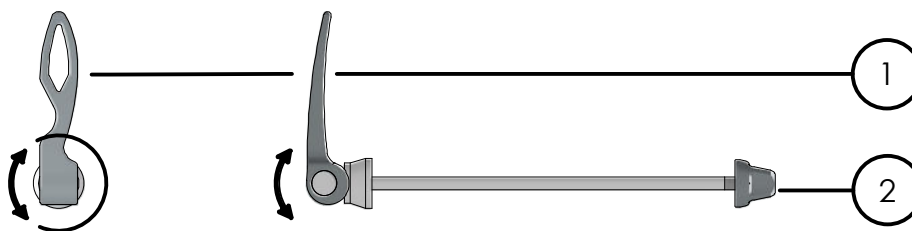
**Closing:** Use the tension-adjusting nut (02) to adjust the tension of the quickrelease mechanism.

Note: The adjustment is correct when you can fully close the lever (01) but with some effort (requiring pressure from the palm of your hand).

Swing the lever (01) 180° up and over to the closed position (the word “CLOSE” will be visible).

Note: If the quick-release lever (01) can be closed using only finger pressure, then the tension-adjusting nut will need to be further tightened.

Caution: The quick-release levers must be firmly tightened before riding the trike.

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### Checking, cleaning and lubricating the chain

Due to the length of the chain on the Kettwiesel, less wear is to be expected compared to conventional bikes. You can check the condition with a chain wear gauge from a bicycle dealer. We recommend that you have this checked by a bicycle mechanic.

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### Maintenance and repair

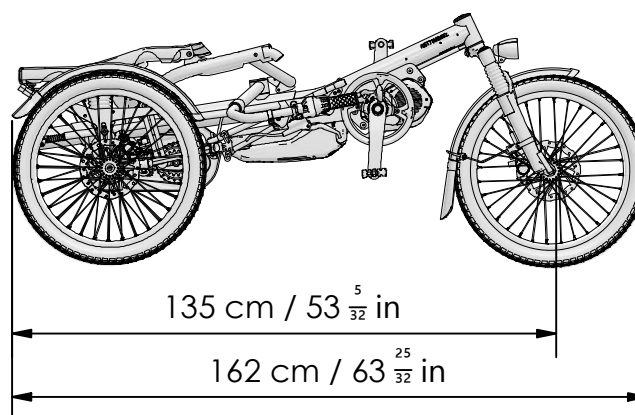
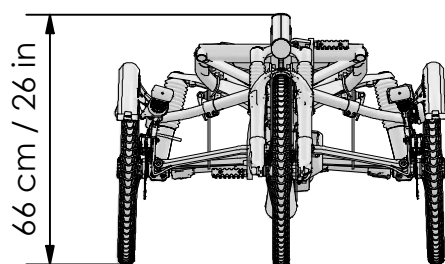
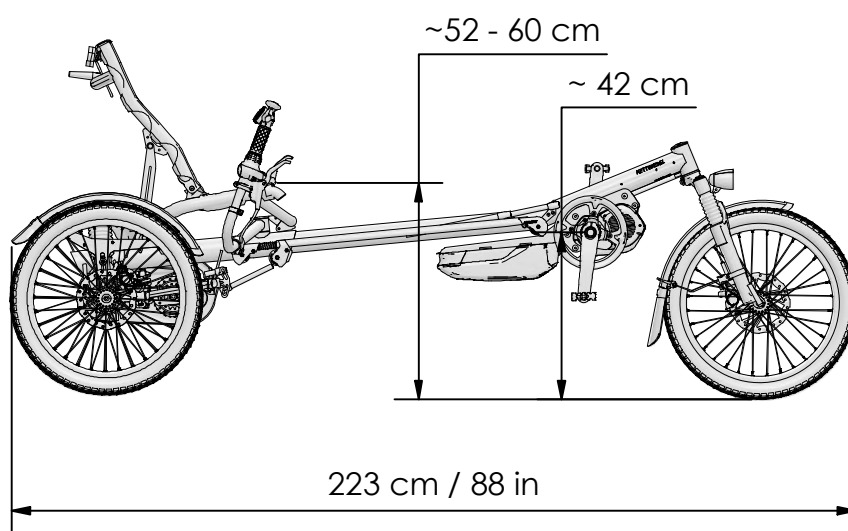
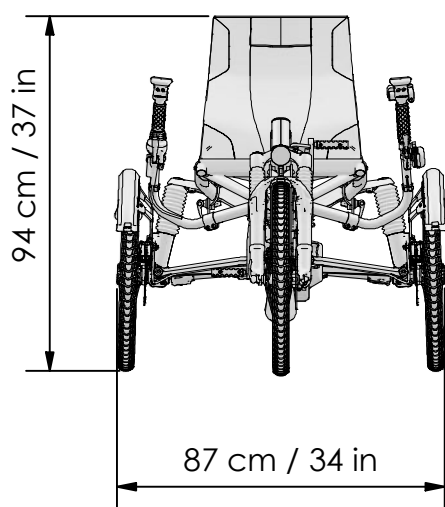
Note: Have all inspection and maintenance work carried out by your HASE Bikes dealer and documented in the inspection pass at the end of these instructions.

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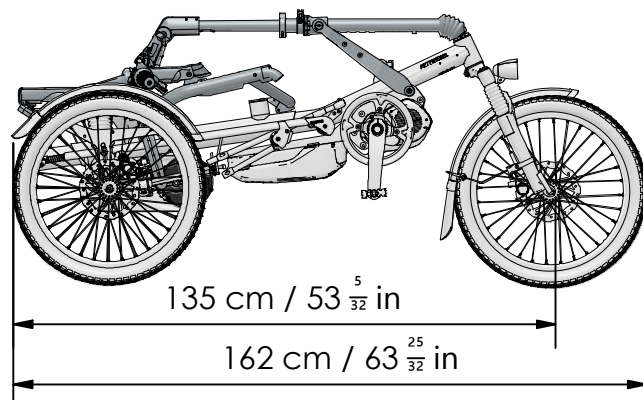
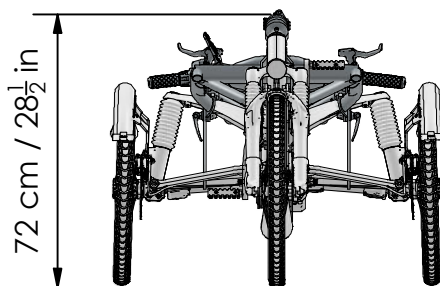
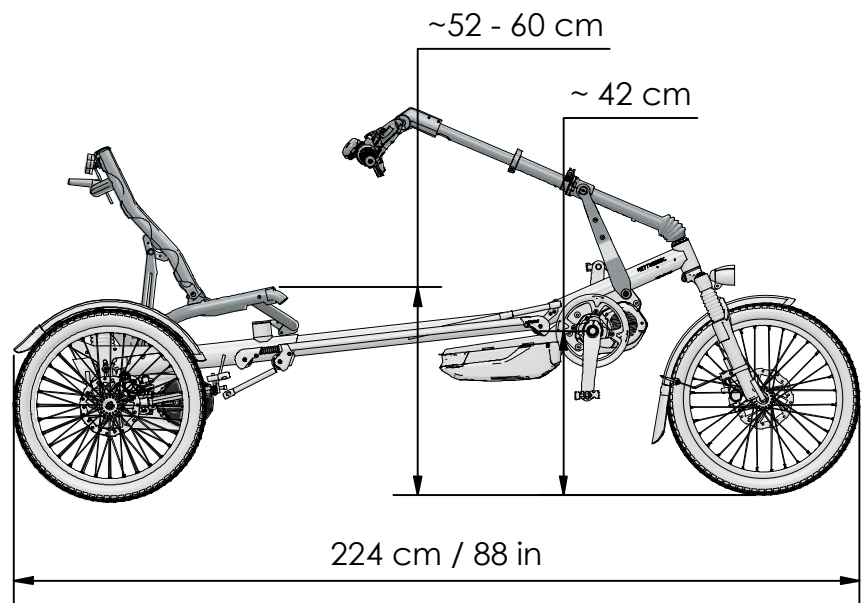
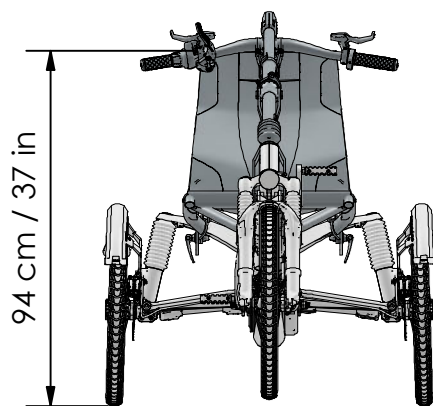
### Kettwiesel ONE (under-seat handlebars):

#### Adjusting the steering

1. Loosen the clamp bolt (1) on the steering rod.
1. Position the front wheel so that it is pointing straight ahead, and brace it in this position.
2. Move the handlebars to the straight-ahead position, and hold them in this position.
4. Tighten the clamp bolt (1) on the steering rod.
5. Check the handlebars' range of movement by moving them to the left and right as far as they will go.

**Caution!** Any restrictions in handlebar movement could lead to accidents with possible injuries.

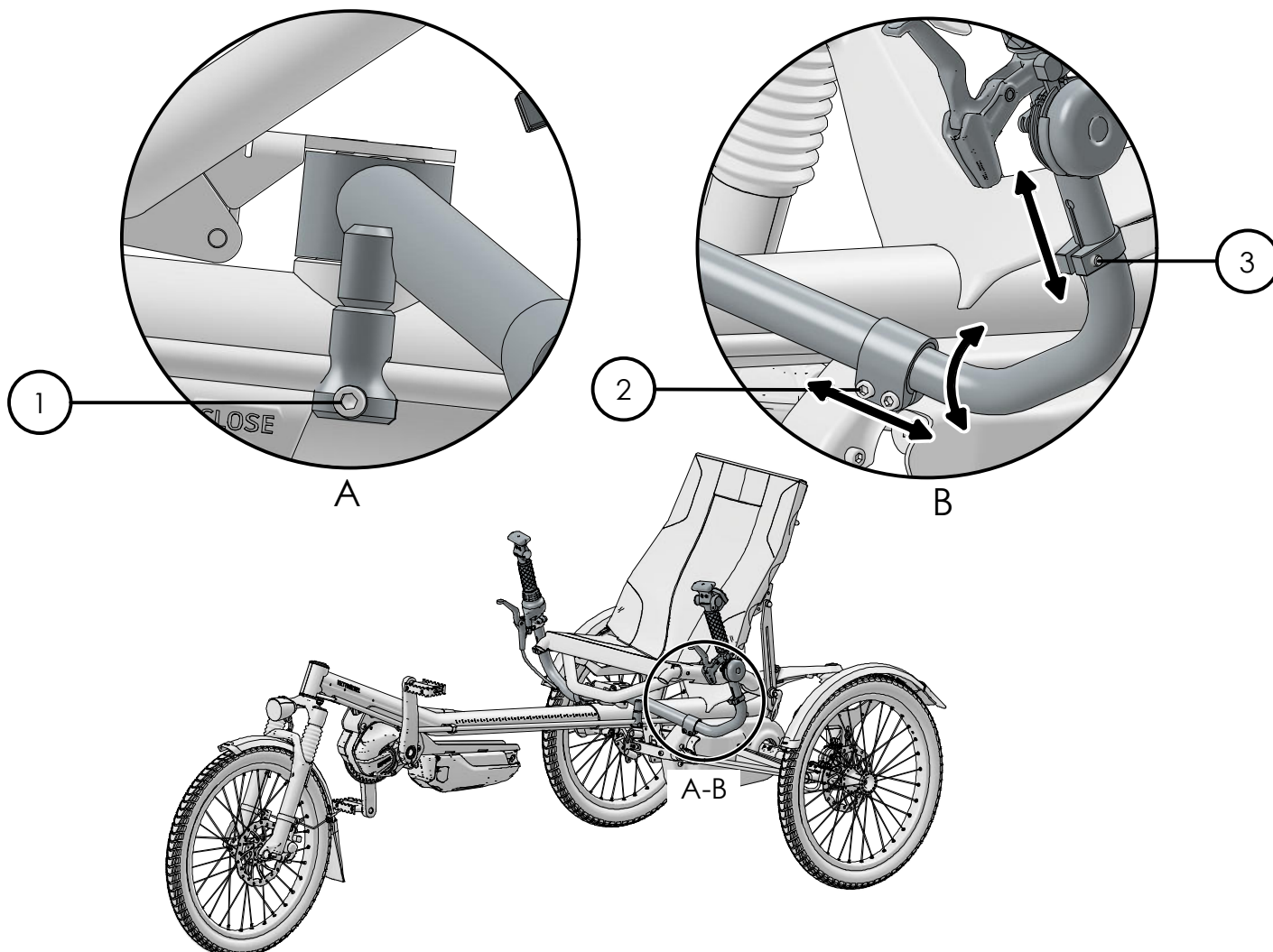
#### Adjusting the handlebar width and angle

1. Loosen the screws (2) on the handlebars.
2. Adjust the handlebar to the required width.
3. Adjust the angle of the handlebars.
4. Retighten the screws (2).

**Caution!** The max. distance from the clamp to the bend of the handlebars should not exceed 80 mm (3 1/8").

#### Adjusting the handlebar height

1. Loosen the screws (3) on the handlebars.
2. Pull the handlebars up/down.
3. Tighten the screws (3).

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### Kettwiesel ONE UP (above-seat handlebars):

#### Adjust handlebar angle

1. Open the lower quick-release (1) on the steering tube.
2. Sit down on the seat, and move the steering tube to the desired angle.
3. Close the lower quick-release (1).

#### Adjusting the handlebar length

1. Open the upper quick-release (2)
2. Press the knob (3) on the lower steering tube, and push or pull the handlebar to the next position.

Note: In an unfavorable setting, the knob (3) can disappear under the guide with the quick release (1).

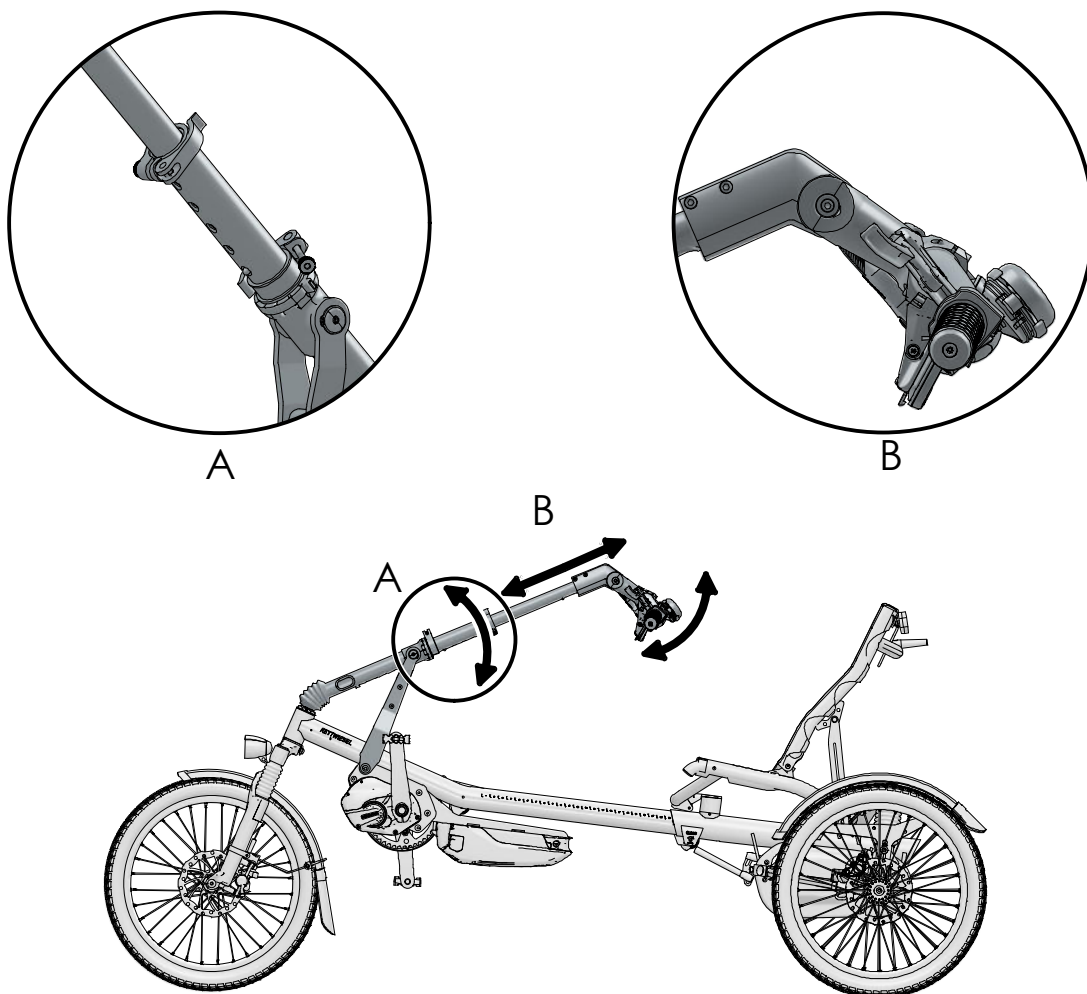
Caution: The length adjustment is not stepless. The knob (3) must be locked into one of the respective holes.

3. Close the quick-release (2).

Note: The clamping force of the fixing sleeve of the upper handlebar can be varied using a cable tie (4). If the fixation is too easy to loosen when the handlebars are in a very upright position, tighten the cable tie (4).

#### Adjusting the angle of the stem

1. Loosen the screw (5) on the stem..
2. Adjust the angle of the stem.
3. Retighten the screw (5).

# Kettwiesel ONE (UP) - An Körpergröße anpassen Kettwiesel ONE (UP) - Adjust to body size

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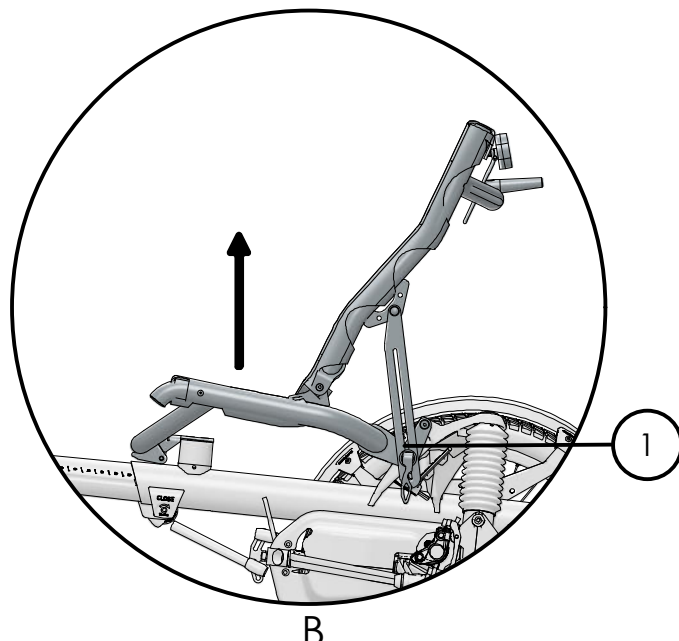
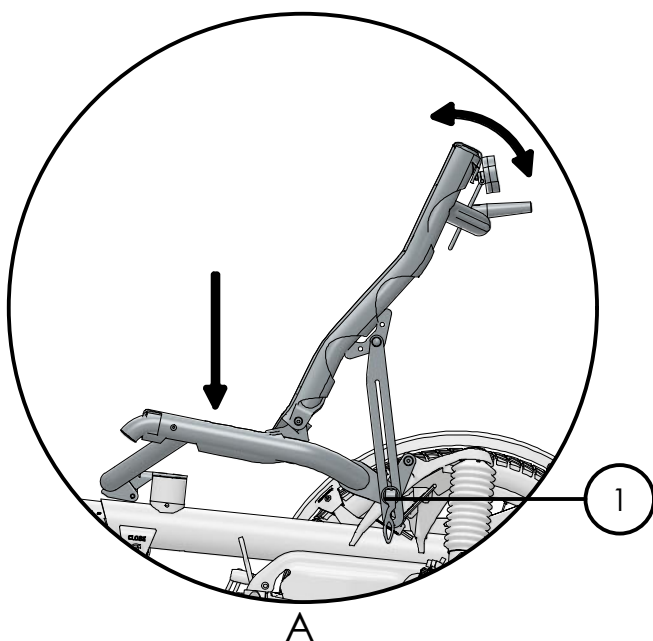
## Adjusting the seat height & angle

Note: Apply the parking brake, to prevent the trike from rolling away.


1. Open the quick-release levers (1) on both sides of the seat.
2. Push the front and rear of the seat down as far as it will go for the low position. Or pull the seat upwards at the front and back for the raised position.

Note: Intermediate positions are not permitted. If the two quick-release levers are open, you can also adjust the angle of the seatback.


3. Close the two quick-release levers (1) on the seat.

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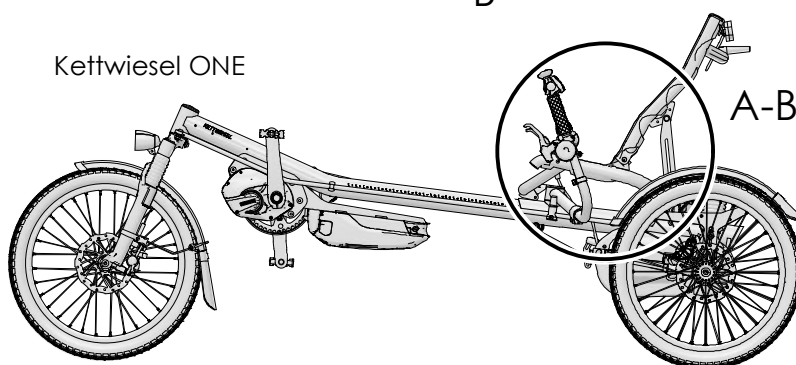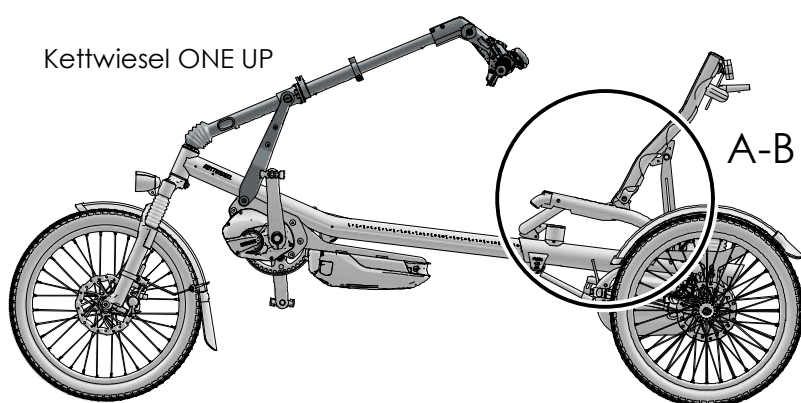### Adjust to body size (under-seat handlebars):

Note: The onboard tool (1) can be found on the lower left side of the main frame. (A)

### Adjust frame length

1. Remove the screw (2) with 4 rotations to release the frame clamp.

Caution! Do not unscrew the screw (2) more than 4 turns. Otherwise the frame may be damaged.

2. Loosen the clamp bolt (3) on the handlebar.
3. Pull the front boom out of the main frame to achieve the desired length.

Caution! The front boom should never be extended past the minimum insertion mark ("STOP"). Riding with an over-extended boom could cause accidents and frame damage. (B)

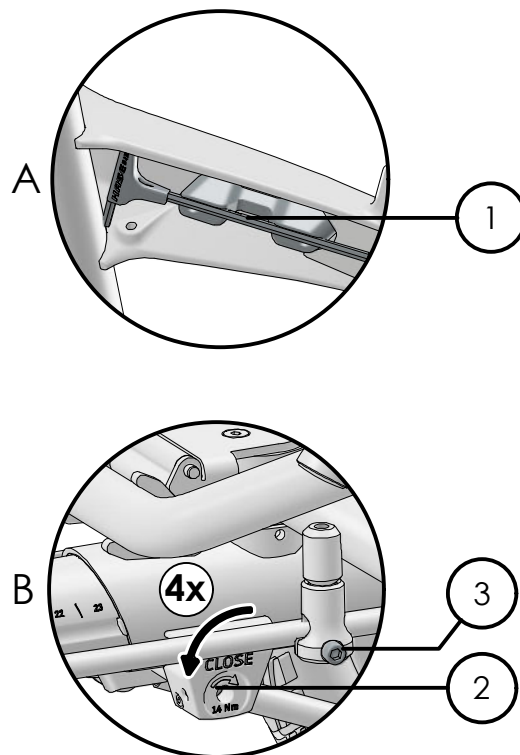
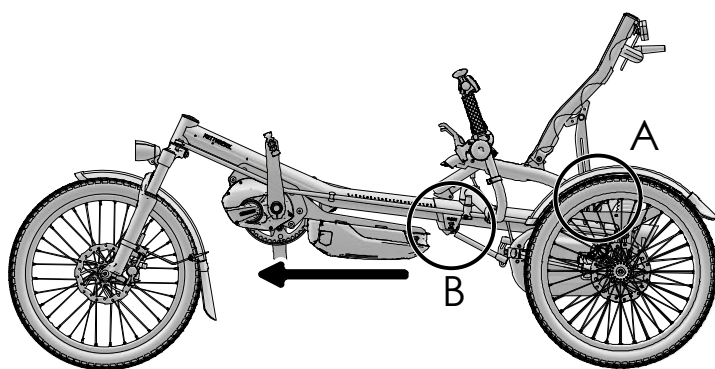
4. Check the seat position.
5. If the position fits, fasten the screw (2) with 4 rotations and tighten it to 14 Nm (10.3 ft-lb).
6. Adjust the steering, and tighten the clamp bolt (3).

### Check seating position

Sit on the seat, and place your feet on the pedals, with the balls of your feet over the center of the axles. When the cranks are in line with the front boom, the front leg should be almost fully extended so that you feel comfortable. If your leg cannot be almost fully extended or it is difficult to reach the pedals with your foot, you will need to adjust the frame length.

### Checking the steering

After changing the frame length of the Kettwiesel ONE, the handlebars must be realigned. See section "steering."

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### Adjust to body size:

Note: The onboard tool (1) can be found on the lower left side of the main frame. (A)

### Adjust frame length

1. Remove the screw (2) with 4 rotations to release the frame clamp.

Caution! Do not unscrew the screw (2) more than 4 turns. Otherwise the frame may be damaged.

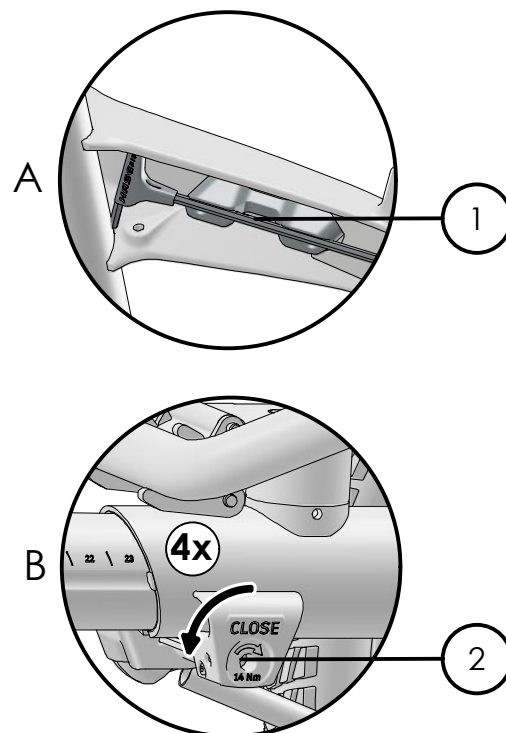
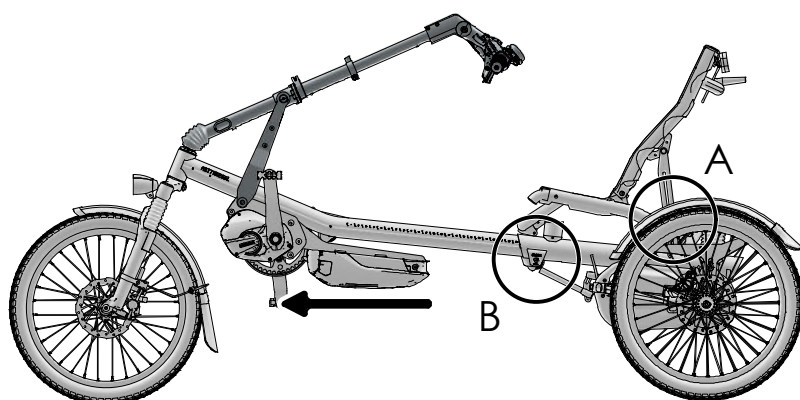
2. Pull the front boom out of the main frame to achieve the desired length.

Caution! The front boom should never be extended past the minimum insertion mark ("STOP"). Riding with an over-extended boom could cause accidents and frame damage. (B)

3. Check the seat position.
4. If the position fits, fasten the screw (2) with 4 rotations and tighten it to 14 Nm (10.3 ft-lb).

### Check seating position

Sit on the seat, and place your feet on the pedals, with the balls of your feet over the center of the axles. When the cranks are in line with the front boom, the front leg should be almost fully extended so that you feel comfortable. If your leg cannot be almost fully extended or it is difficult to reach the pedals with your foot, you will need to adjust the frame length.

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

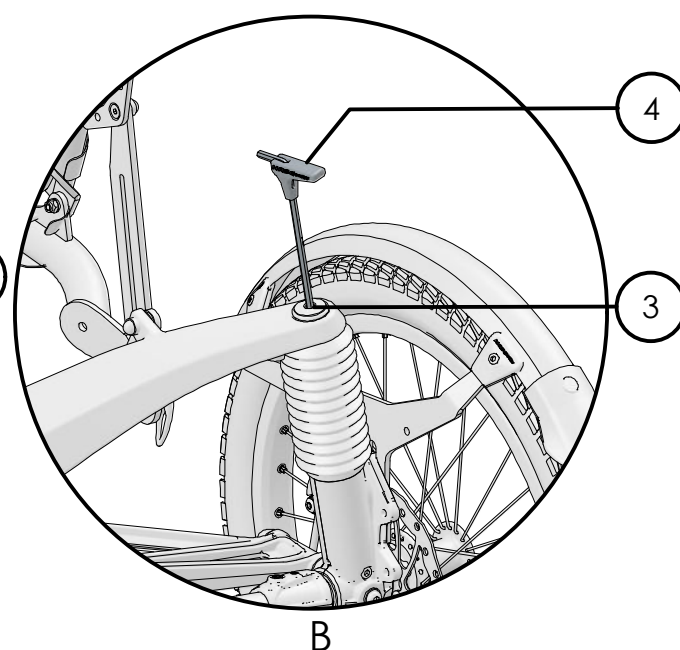
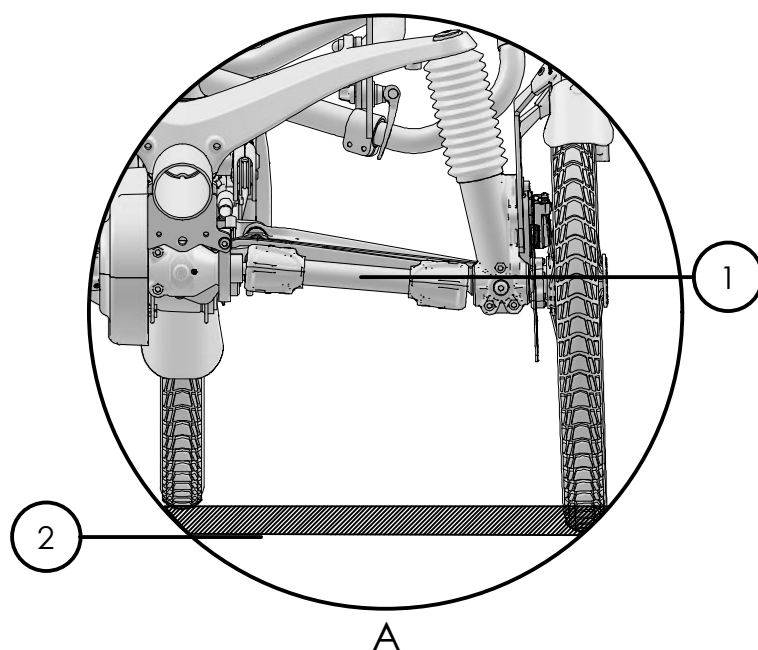
1. Sit on the seat. If the suspension is properly adjusted, the drive shafts (1) will be parallel to the road surface (2).

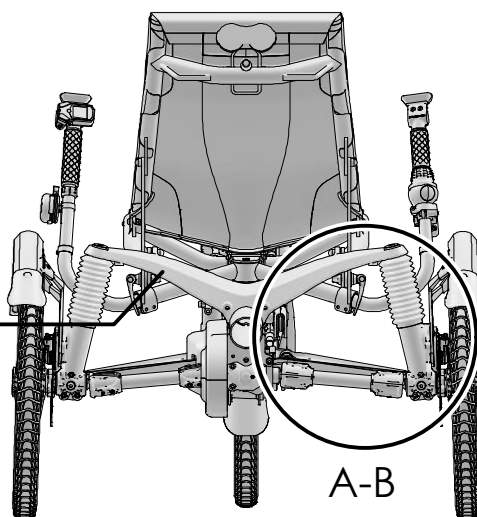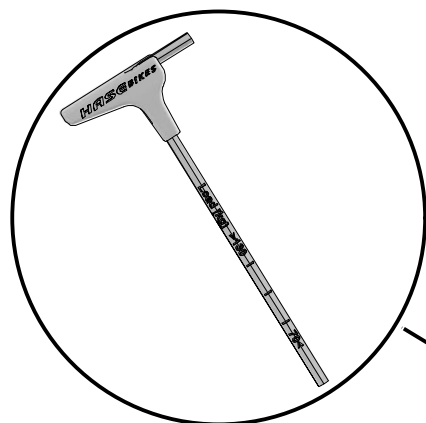2. If the outer ends of the drive shafts are tilted upwards, you should increase the preload of the suspension. To do so, use the onboard tool (4) to tighten the adjusting screw (3). If the outer ends of the drive shafts (1) are tilted downwards, the preload is too high. Loosen the adjusting screw (3).

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### Parking brake

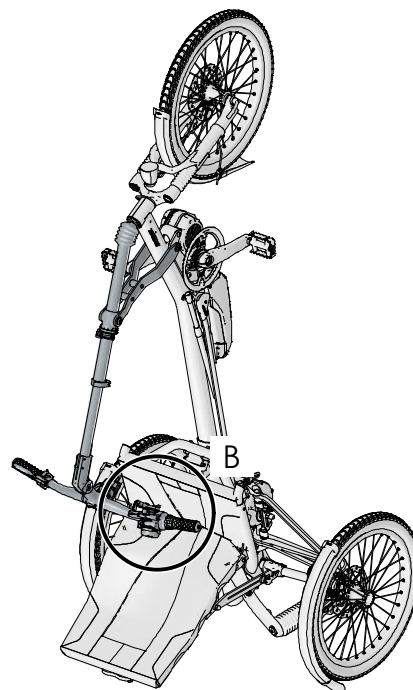
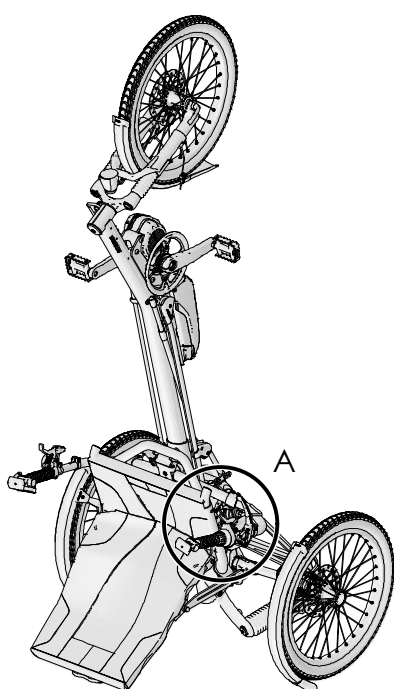
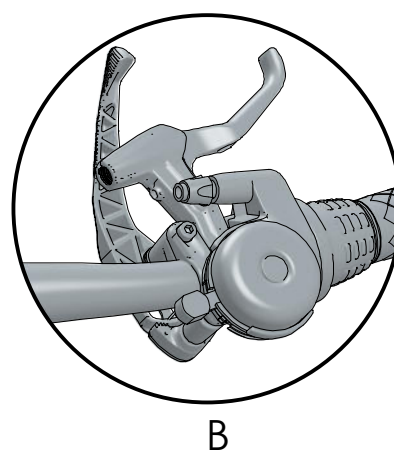
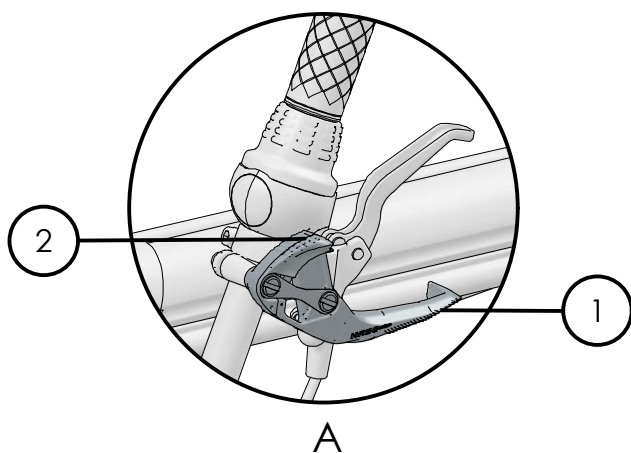
To prevent the parked Kettwiesel from rolling away, it is equipped with a parking brake. The parking brake lever is located on the handlebar grip.

1. Pull the parking brake lever (1) until the desired braking effect is achieved.
2. To release the parking brake, press the release lever (2).

### Vertical parking

The Kettwiesel can be parked upright with the front wheel in the air.

1. Engage the parking brake.
2. To stand the Kettwiesel upright, pull it up by the front wheel until it is resting securely on the seatback and rear wheels.

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## Warranty

The statutory warranty applies. The dealer must sell you the HASE BIKE in a safe and roadworthy condition. In addition, the company Hase Bikes offers a warranty of three years - from the date of purchase - against frame breakage in accordance with the following conditions. If the bike is registered at <https://hasebikes.com/de/formulare/fahrradpass/>, this warranty is extended to 5 years.

- HASE BIKES is only liable for defects caused by faulty workmanship on the frame or in the assembly process. Components are covered by the respective manufacturer's warranty.

- The additional warranty from HASE BIKES only applies to the original owner and does not cover damage resulting from failure to observe the information in this user's manual.

- Within the warranty period of 3 years from the date of purchase, HASE BIKES will repair any broken frame part free of charge. If the part cannot be repaired, it will be replaced free of charge with a new or like-new part. Replaced parts become the property of HASE BIKES.

- Any further claims beyond the scope of this warranty are excluded. In particular, the company HASE BIKES will not cover the costs of removing or installing parts (e.g. by the dealer) or shipping costs. We do not accept postal deliveries with insufficient postage.

- The statutory and HASE BIKES warranty will immediately become void if the Kettwiesel is used for competition purposes, ridden on rough terrain, or used in any manner considered abnormal for a bicycle

- Damage resulting from accidents is not covered by the warranty.

- The warranty also becomes void if the Kettwiesel is fitted with incompatible accessories or if accessories are installed incorrectly. Therefore, if you need to replace or install any components, be sure to have the associated work performed by an authorized dealer.

- The warranty becomes void if any improper modifications (e.g. grinding, drilling, bending, etc.) are made to the frame.

- Any repairs or replacements made without charge during the warranty period do not carry a new warranty beyond that of the original purchase.

- A warranty claim can only be processed through the authorized HASE BIKES dealer from whom the bike was purchased or through one of our distribution partners.

## Symbols:

-  – CE conformity mark
-  – Product manufacturer
-  – Follow instructions for use
-  – Item number of the manufacturer
-  – Store dry
-  – Medicine Product
-  – Batch number
-  – Serial number

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Have your trike checked by your authorized Hase Bikes dealer at regular intervals. To avoid unnecessary downtimes and waiting periods, it is advisable to have the service carried out in the fall or winter months, or to make an appointment in advance during the high season.

|                   |  |
|-------------------|--|
| Frame number:     |  |
| Frame color:      |  |
| Date of purchase: |  |
| Special features: |  |

|                                                                                         |                          |
|-----------------------------------------------------------------------------------------|--------------------------|
| 1st service: After 300km (200miles) or three months after date of purchase: Job number: | Date:                    |
| Replaced or added parts:                                                                | Dealer's signature/stamp |
|                                                                                         |                          |
|                                                                                         |                          |
|                                                                                         |                          |
|                                                                                         |                          |

|                                                                                          |                          |
|------------------------------------------------------------------------------------------|--------------------------|
| 2nd service: After 2000km (1250 miles) or 1 year after the date of purchase: Job number: | Date:                    |
| Replaced or added parts:                                                                 | Dealer's signature/stamp |
|                                                                                          |                          |
|                                                                                          |                          |
|                                                                                          |                          |
|                                                                                          |                          |

|                                                                                           |                          |
|-------------------------------------------------------------------------------------------|--------------------------|
| 3rd service: After 4000km (2500 miles) or 2 years after the date of purchase: Job number: | Date:                    |
| Replaced or added parts:                                                                  | Dealer's signature/stamp |
|                                                                                           |                          |
|                                                                                           |                          |
|                                                                                           |                          |
|                                                                                           |                          |

|                                                                                              |                          |
|----------------------------------------------------------------------------------------------|--------------------------|
| 4th inspection: After 6000km (3750 miles) or 3 years after the date of purchase: Job number: | Date:                    |
| Replaced or added parts:                                                                     | Dealer's signature/stamp |
|                                                                                              |                          |
|                                                                                              |                          |
|                                                                                              |                          |
|                                                                                              |                          |

|                                                                                           |                          |
|-------------------------------------------------------------------------------------------|--------------------------|
| 5th service: After 8000km (5000 miles) or 4 years after the date of purchase: Job number: | Date:                    |
| Replaced or added parts:                                                                  | Dealer's signature/stamp |
|                                                                                           |                          |
|                                                                                           |                          |
|                                                                                           |                          |
|                                                                                           |                          |

|                                                                                            |                          |
|--------------------------------------------------------------------------------------------|--------------------------|
| 6th service: After 10000km (6250 miles) or 5 years after the date of purchase: Job number: | Date:                    |
| Replaced or added parts:                                                                   | Dealer's signature/stamp |
|                                                                                            |                          |
|                                                                                            |                          |
|                                                                                            |                          |
|                                                                                            |                          |